



Apache Log4j Vulnerabilities - LATITUDE™

Boston Scientific has reviewed the Apache Log4j Vulnerabilities impacting Apache Log4j versions 2.17 and older for the LATITUDE™ product group. See below for the outcome of that investigation:

Product	Model Number	Status
LATITUDE™ NXT	6446, 6447, 6448, 6449, 6460	Not vulnerable
LATITUDE Clarity™	7260	Not vulnerable
myLUX™ Patient App	2925, 6259	Not vulnerable
LUX-Dx™ Clinic Assistant App	2935, 6256	Not vulnerable
LATITUDE Consult™	6297	Not vulnerable
LATITUDE™ EMR Secure Courier Client	6455, 6624	Not vulnerable Vendor INFOR states product is not affected.
LATITUDE Link™	6215	Customer action needed to secure customer system. Reference the mitigation instructions below if you are using LATITUDE Link
LATITUDE™ Classic	6488	Not vulnerable
MyLATITUDE™ Patient App	N/A	Not vulnerable
LATITUDE™ Programmer & Apps	3300	Not vulnerable
ZOOM™ Programmer & Apps	3120	Not vulnerable
LATITUDE™ Communicators	6290, 6280, 6288, 6299, 6476, 6496, 6468, 6420, 6498, 6443	Not vulnerable
Heart Connect™ Tablet	3933	Not vulnerable

Visit www.bostonscientific.com/product-security for the latest official updates and additional details.

For More Information:

United States Technical Services
1.800.CARDIAC (227.3422)
tech.services@bsci.com

International Technical Services
+32 2 416 7222
intltechservice@bsci.com

Asia Pacific Technical Services
+61 2 8063 8299
aptechservice@bsci.com



LATITUDE Link™ Mitigation for Apache Log4j Vulnerabilities

Technical Details

The LATITUDE Link Viewer portion of the Link application uses Log4j. A user's computer running LATITUDE Link is susceptible to the vulnerability.

To mitigate the risk this vulnerability poses, please see the mitigation detailed below.

Mitigation

The following mitigation **prevents** a successful exploitation:

Upgrade to LATITUDE Link version 1.7.

- a. LATITUDE Link version 1.7 will be available on January 26, 2022.
- b. Once LATITUDE Link Viewer 1.7 is available, a yellow banner notification will be displayed in the LATITUDE Link Viewer application until the software update is complete.
- c. To run the LATITUDE Link Updater on the machine where the LATITUDE Link Viewer is installed, your IT administrator can navigate: Start Menu > Boston Scientific > LATITUDE Link Viewer Updater.
- d. For assistance, contact the LATITUDE EMR Integration Support team at 888-283-8713.

Resources

If assistance is required to implement one of the above mitigations, have your IT department contact the LATITUDE EMR Support line at 888-283-8713.

Revision 3 – January 25, 2022