

SpeedBand SuperView Super 7™



Multiple Band Ligator

Boston
Scientific



Attach Handle Unit to Endoscope

- 1- Step One** Straighten endoscope
- 2- Step Two** Feed trip wire through working channel
- 3- Step Three** Insert handle unit into biopsy valve
- 4- Step Four** Secure handle to endoscope with strap

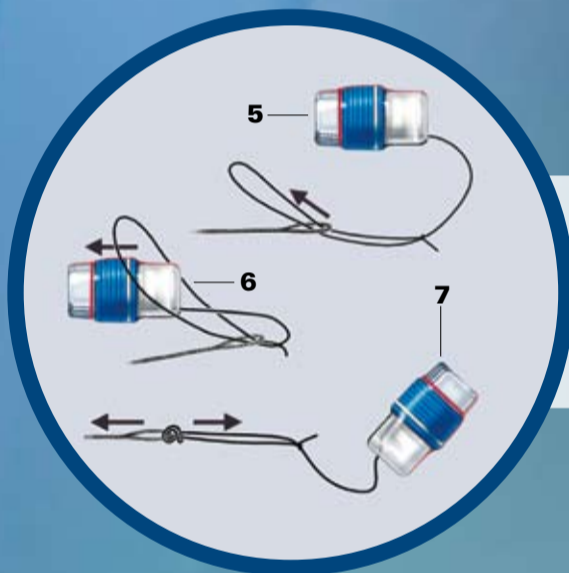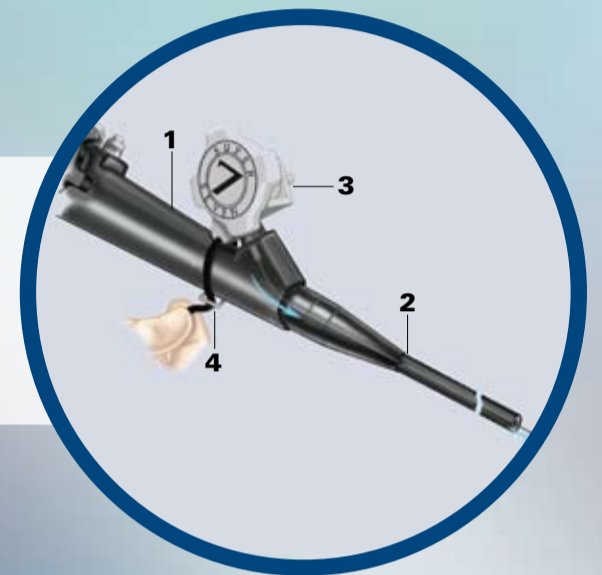


Attach Ligating Unit to Trip Wire

- 5- Step Five** Thread pulling loop through trip wire loop
- 6- Step Six** Thread ligating unit through pulling loop
- 7- Step Seven** Pull to tighten

Attach Ligating Unit to Endoscope

- 8- Step Eight** Grasp trip wire loop at handle and retract until ligating unit contacts tip of endoscope
- 9- Step Nine** Align black stripe with working channel and push ligating unit onto endoscope
DO NOT TAKE UP TRIP WIRE SLACK BY ROTATING HANDLE UNIT

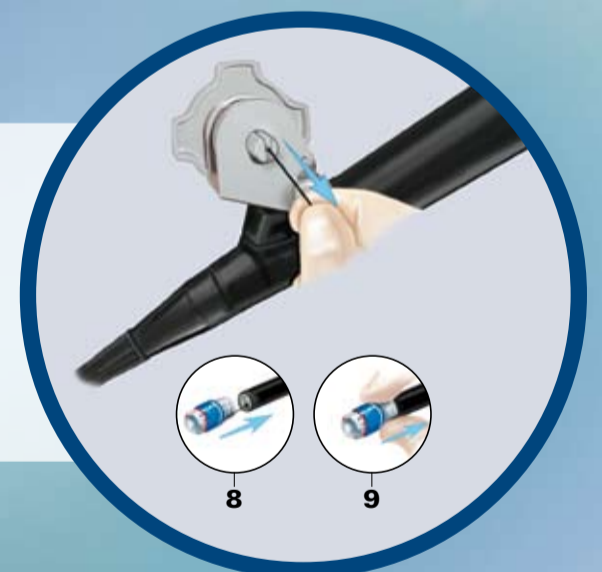

Complete Preparation for Use

- 10- Step Ten** Grasp trip wire and retract to remove slack, then lock down by pulling across narrow slot
DO NOT OVER TIGHTEN
- 11- Step Eleven** Carefully remove shrink wrap by hand
- 12- Step Twelve** OPTIONAL: Attach irrigation tube

Boston
Scientific

Delivering what's next.™

www.bostonscientific.com
www.bostonscientific-international.com

RCS Nanterre B420 668 420
© 2007 Boston Scientific Corporation
or its affiliates. All rights reserved.
DINEND2168EB

Preparation For Use*

*This overview is provided for illustrative purposes only and is intended only as a brief summary of how the set-up procedures for the SpeedBand SuperView Super 7 are generally performed. Please refer to the Directions For Use for complete set-up instructions and precautions. NOTE: Designed to fit on scopes with outer diameter of 8.6–11.5mm. All cited trademarks are the property of their respective owners. CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labelling supplied with each device. Information for the use only in countries with applicable health authority product registrations.

PSST IC 3905A Printed in France by cwcreativeservices.