



Refer to the device directions for use for complete instructions on device use.

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

Reuse Warning

For single use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure. Guidewire reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

Intended Use/ Indications For Use

The SpyGlass Discover Jagwire Guidewire is indicated to provide access to the pancreaticobiliary system. The Guidewire is designed to be used for catheter introduction, exchanges of catheters and to aid in the placement of diagnostic and therapeutic devices.

Contraindications

None known.

Precautions

- Do not use a guidewire which has been damaged.
- Damage will prevent a guidewire from performing with accurate torque or tactile response and may cause injury. Return any defective product to Boston Scientific for replacement.
- The Guidewire should only be used by physicians thoroughly trained in pancreaticobiliary procedures.

Adverse Events

Complications which can result from the use of guidewires in pancreaticobiliary applications include, but are not limited to:

- Perforation
- Hemorrhage
- Edema
- Bleeding
- Tissue trauma
- Infection