

RANGER™ SFA Trial

12-month results presented at Charing Cross 2017

Objective To prove the superior performance of the Ranger™ paclitaxel-coated PTA balloon catheter (Boston Scientific) for angioplasty for femoropopliteal artery lesions when compared to non-coated balloons.

Trial Design Prospective, multicenter, randomized, controlled trial (2:1 Ranger DCB vs. non-drug-coated balloon). Follow up through 3 years.


- Key Enrollment Criteria**
- Rutherford 2, 3 or 4
 - Stenotic, restenotic or occlusive lesions ($\geq 70\%$ stenosis) in the native non-stented SFA/PPA
 - No prior treatment with drug coated balloons or drug-eluting stents in the treated limb
 - Lesion length ≥ 20 mm and ≤ 150 mm

Key Baseline Lesion Characteristics#
#core lab adjudicated

	PTA (34)	Ranger DCB (71)
Target Lesion Length (mm)	60 ± 48	68 ± 46
Reference Vessel Diameter (mm)	4.5 ± 0.83	5 ± 0.89
Percent Diameter Stenosis (%)	82 ± 18	85 ± 15
Total Occlusions (%)	34	34


12 Month Outcome

- Patency**
- Ranger achieved an 86% primary patency at 12 months (patent by DUS and without reintervention)
 - Ranger demonstrated primary patency superior* to PTA

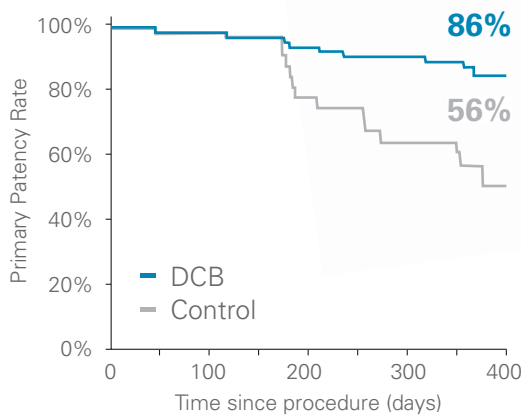

*p<0.001 log rank test, Kaplan-Meier Analysis

- Freedom from TLR**
- Ranger achieved a 91% Freedom from TLR at 12 months
 - Ranger demonstrated Freedom from TLR superior* to PTA

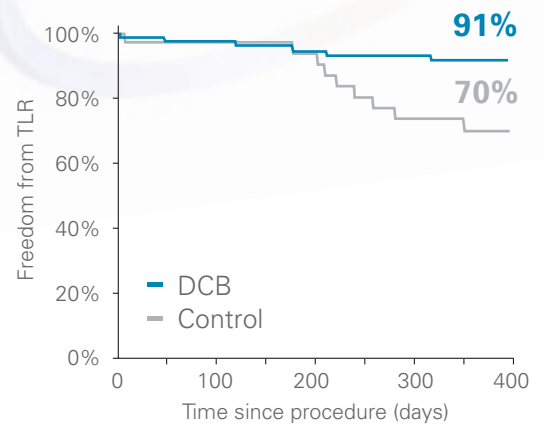


*p=0.010 log rank test, Kaplan-Meier Analysis

RANGER™ SFA Trial

12-month results presented at Charing Cross 2017


Primary Patency at 12 months from DCB trials of the SFA

^aKM Estimates

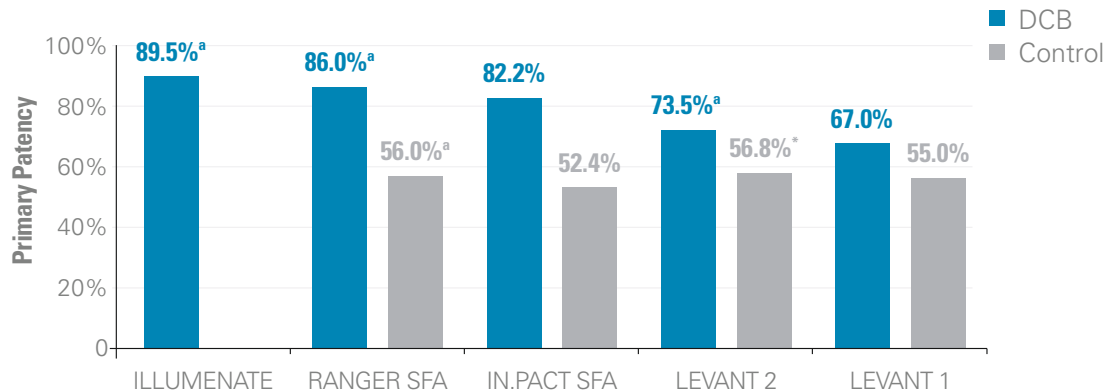

LEVANT 2 - Rosenfield K, et al. N Engl J Med. 2015 9;373(2):145-53., ILLUMENATE - Schroeder H, et al. Catheter Cardiovasc Interv. 2015;86(2):278-86., LEVANT 1 - Scheinert D, et al. JACC Cardiovasc Interv. 2014;7(1):10-19., IN.PACT SFA - Tepe G, et al. Circulation. 2014 pii: CIRCULATIONAHA.114.011004.

TLR at 12 months from DCB trials of the SFA

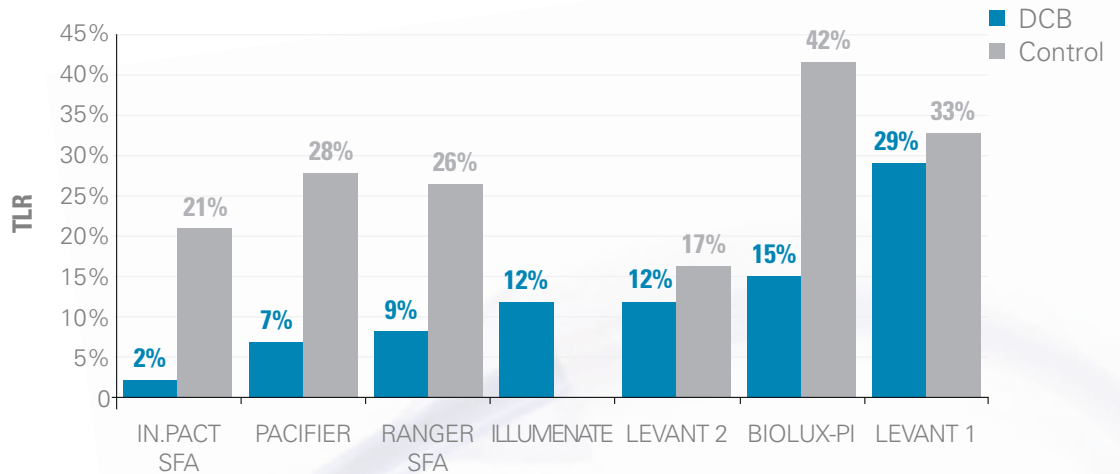

BIOLUX-PI - Scheinert D, et al. J Endovasc Ther 2015; 22(1): 14-21., LEVANT 2 - Rosenfield K, et al. N Engl J Med. 2015 9;373(2):145-53., PACIFIER - Werk M, et al. Circ Cardiovasc Interv. 2012;5(6):831-840., ILLUMENATE - Schroeder H, et al. Catheter Cardiovasc Interv. 2015;86(2):278-86., LEVANT 1 - Scheinert D, et al. JACC Cardiovasc Interv. 2014;7(1):10-19., IN.PACT SFA - Tepe G, et al. Circulation. 2014 pii: CIRCULATIONAHA.114.011004.

Patient Outcomes

Rutherford score

- 84% of Ranger subjects had no or mild symptoms - Rutherford category 0 or 1
- Compared to baseline, Ranger and PTA groups both showed improvement in Rutherford score and ABI

Conclusions

- Greater patency rate at 12 months for Ranger DCB than Control (86% vs 56%)
- Freedom from TLR greater for Ranger DCB than Control at 12 months (91% vs 70%)
- Patients treated with Ranger DCB demonstrated significant improvements in symptoms and hemodynamics at 12 months
 - Symptomatic improvement generally similar to Control but with ~1/3 as many revascularizations

Ranger™ Paclitaxel Coated PTA Balloon Catheter is manufactured by Hemoteq AG and distributed by Boston Scientific Corporation. All cited trademarks are the property of their respective owners. CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labeling supplied with each device. Information for the use only in countries with applicable health authority product registrations. Material not intended for use or distribution in France.