



The ACURATE *neo2*™ ComAlign 4-step Guide

1



Down at
6 o'clock


Position handle with **safety button**
at 6 o'clock, facing down

2



3 Cusp View

(ACURATE *neo2* marker at top of pigtail)

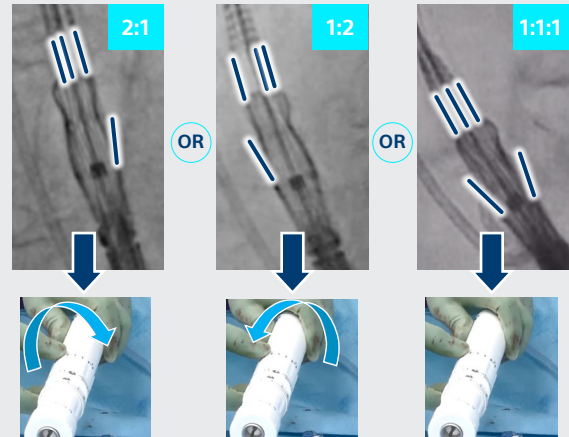

Rotate front part of the handle until two free
cells are symmetrical visible in 3 cusp view
Most cases alignment achieved between
0.5 and 1.5 handle rotations (180° to 540°)

3



Cusp Overlap




In cusp overlap **one free cell** should
be on the inner curvature
If incorrect – Rotate either CW or
CCW and revert to step 2

4




3 Cusp View

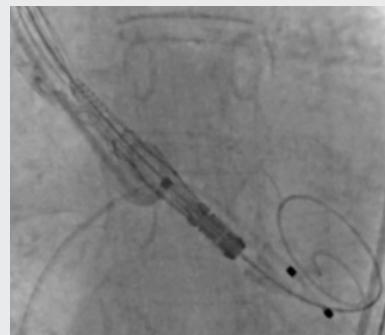

After alignment, allow handle
to **rotate back slowly** in cradled hands
Proceed with ACURATE *neo2*
implantation steps

Assessment & Reminders

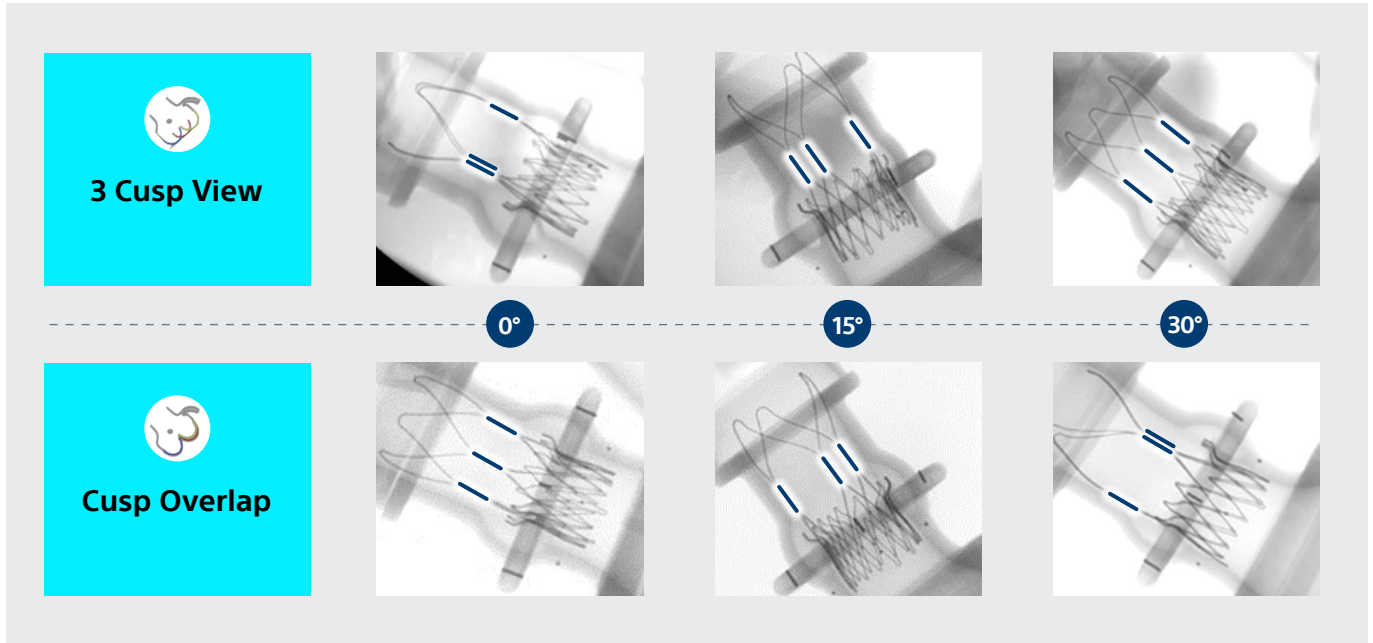


- › Perform alignment with positioning marker at **top of pigtail**
- › **Slowly** twist handle 180° before valve starts to rotate
Most cases achieve alignment between 0.5 and 1.5 handle rotations (180° to 540°)
- › Tortuosity increases torque required
- › Care required on where you hold handle during torque to avoid accidental release
- › Dissipating the torque is important to keep position once it is achieved
- › **Deployment** should be done in a 3 cusp view as normal

Easy check of alignment using posts and both views

- › 0°-15° Commissural alignment
- › 15°-30° Mild commissural misalignment
- › 30°-45° Moderate commissural misalignment
- › 45°-60° Severe commissural misalignment

Immerse into a 3D commissural alignment experience

An advanced & realistic tool enabling you to experience & practice the commissural alignment technique with ACURATE *neo2*.

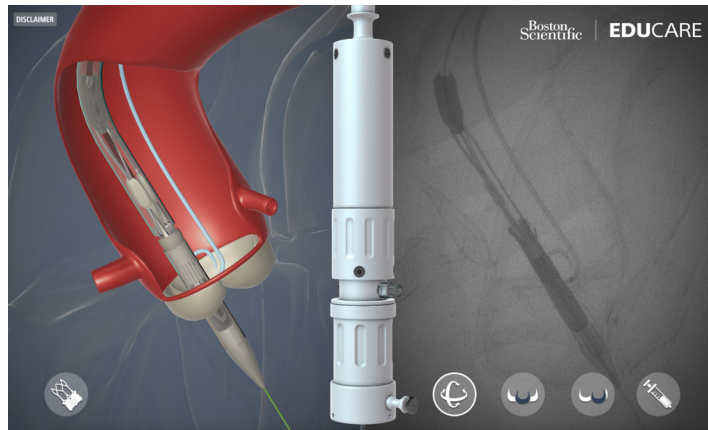

[Try it now](#)

Continue your learning journey on commissural alignment

Discover these useful courses from experts focusing on Commissural alignment and Coronary access.

Commissural Alignment & Coronary Access Series

[Learn now](#)

| Expert | Affiliation |
|----------------------|--|
| Dr. Henrik Bjursten | Skåne University Hospital Lund, Sweden |
| Dr. Matthias Gotberg | Skåne University Hospital Lund, Sweden |
| Prof. Ole de Backer | Rigshospitalet Copenhagen, Denmark |
| Dr. Darren Mylotte | University Hospital Galway, Ireland |
| Mohamed Abdel-Wahab | Leipzig Heart Center Leipzig, Germany |

[Get Full Access now](#)

* De Backer O. Commissural alignment & TAV performance: Results from the COMALIGN-neo 2 study. TCT 2023

1. TVT Registry.
2. Faroux L, et al. ST-Segment Elevation Myocardial Infarction Following Transcatheter Aortic Valve Replacement. "https://www.jacc.org/journal/jacc" J Am Coll Cardiol. 2021 May, 77 (17) 2187 -2199.
3. Bieliauskas, G, De Backer, O, Søndergaard, L, et al. Patient-Specific Implantation Technique to Obtain Neo-Commissural Alignment with Self-Expanding Transcatheter Aortic Valves. J Am Coll Cardiol 2021.

Illustrations for educational purposes only, not indicative of actual size or performance.
All photographs taken by Boston Scientific.

All trademarks are property of their respective owner. CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings, and instructions for use can be found in the product labelling supplied with each device or at www.IFU-BSCI.com. Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries. This material not intended for use in France.

Boston Scientific
Advancing science for life™

www.bostonscientific.eu

© 2024 Boston Scientific Corporation or its affiliates. All rights reserved.
SH-1864404-AA