



Refer to the device directions for use for complete instructions on device use.

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

Warning

Contents supplied STERILE using an ethylene oxide (EO) process. Do not use if sterile barrier is damaged. If damage is found, call your Boston Scientific representative.

For single 24 hour use only. Do not reprocess or resterilize. Reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

Within 24 hours, dispose of product and packaging in accordance with hospital, administrative and/or local government policy.

Intended Use/ Indications For Use

The Hydra Water Bottle Cap System is intended to be used with an air or CO2 source for the purpose of supplying sterile water to the endoscope during endoscopic procedures. It is compatible with commercially available sterile water bottles.

Contraindications

Package contents include:

- (1) Water Bottle Cap System

Adverse Events

Potential contamination