



# Pulsed Electric Field versus Cryoballoon to Treat Paroxysmal Atrial Fibrillation (PERFECT-PAF) Randomised Trial: A Periprocedural Clinical and Cost Analysis


Chierchia, et al. ESC Sept 2, 2024

## OBJECTIVE


- ▶ The PERFECT-PAF trial was designed to compare the safety, efficacy and procedural costs of FARAPULSE™ Pulsed Field Ablation versus cryoballoon ablation (CBA) for pulmonary vein isolation in paroxysmal atrial fibrillation (PAF) patients.



SAFETY

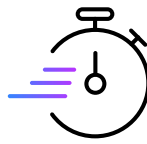


EFFICIENCY

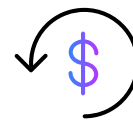

COST

## METHODS

- ▶ The clinical safety and economic value of FARAPULSE versus CBA was assessed in a multicentre, randomised, controlled, non-inferiority trial.
- ▶ Data was collected across 6 European centres between October 2023 and May 2024.
- ▶ The primary safety endpoint included procedure and device related adverse events (AEs) within 7 days post-ablation.
- ▶ Procedural parameters [procedure times, anesthesia, complications] were collected on FARAPULSE (n = 134) and CBA (n = 135) patients in addition to clinical management and health care utilisation data ≤30 post procedure [length of stay, ICU admission, emergency (ED) and outpatient visits, and rehospitalisations].
- ▶ Total costs were calculated based on the cost of the procedure, anaesthesia, re-hospitalisation, ICU, ED and outpatient visit costs.

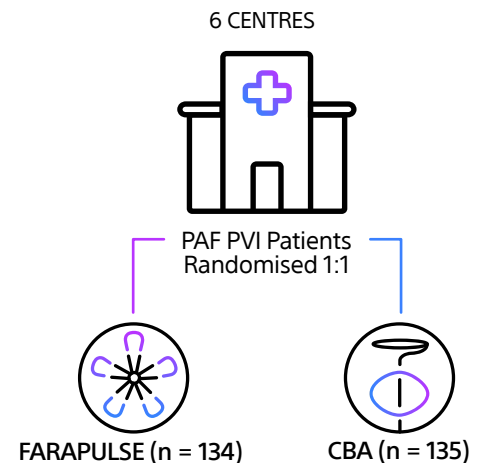

## RESULTS

### SAFETY

- ▶ The incidence of adverse events both during the procedure and within a 7-day period was significantly reduced with FARAPULSE, at 0.7%, compared to CBA's 8.1% (p = 0.0053), and 0.7% against 5.9% (p = 0.036) respectively.



SAFETY

**7-day Adverse Event Rate**

**0.7%\*** vs.

**5.9%**

FARAPULSE

\*significantly lower p = 0.036

CBA

	PFA (n = 134)	CRYO (n = 135)	p-value
Overall periprocedural adverse events, n (%)	1 (0.7%)	11 (8.1%)	0.0053
Major adverse events, n (%)	1 (0.7%)	4 (3.0%)	0.3703
Femoral artery pseudoaneurysm*	1 (0.7%)	1 (0.7%)	1.000
Tamponade requiring drainage	0 (0.0%)	1 (0.7%)	1.000
Persistent phrenic nerve injury	0 (0.0%)	2 (1.5%)	0.4981
7-day adverse event rate, n (%)	1 (0.7%)	8 (5.9%)	0.0360

\*requiring surgery

## PROCEDURAL CHARACTERISTICS

- FARAPULSE had significantly lower ( $p < 0.001$ ) room time, procedure time, LA dwell time and fluoroscopy times.

	PFA	CBA	Time Savings
Room Time	77 ± 14	97 ± 20	20 min
Procedure Time	53 ± 12	69 ± 18	17 min
LA Dwell Time	28 ± 8	48 ± 15	20 min
Fluoroscopy	12 ± 5	15 ± 6	3 min

Mean ± SD

More predictable  
procedures with  
**FARAPULSE**

\*\*Lower standard deviations – F-statistic analysis



EFFICIENCY

## COST PER PATIENT

- The total costs per patient\* within 30-days were 13.7% lower with FARAPULSE.

\*Product material is not included in the cost

- The driver of the cost savings with FARAPULSE was the significantly shorter procedure times ( $p < 0.001$ ). Length of hospital stay, ICU, ED usage, outpatient visits, and re-hospitalisations within 30-days of the intervention did not differ between FARAPULSE and CBA.



COST

**13.7%** lower with  
**FARAPULSE**  
due to  
lab efficiency

## CONCLUSIONS

- FARAPULSE had fewer overall peri-procedural and 7-day adverse event rates compared to CBA in PAF patients.
- FARAPULSE procedure times were shorter and more predictable resulting in an overall lower cost per patient at 30-days due to lab efficiency.
- Long-term follow-up data will be analysed to further investigate the potential clinical and economic impact of FARAPULSE vs. CBA.

Chierchia G., et al. Pulsed Electric Field versus Cryoballoon to Treat Paroxysmal Atrial Fibrillation (PERFECT-PAF) Randomized Trial: A Periprocedural Clinical and Cost Analysis. ESC, Sept 2, 2024.

**Boston  
Scientific**  
Advancing science for life™

[www.bostonscientific.eu](http://www.bostonscientific.eu)

© 2024 Boston Scientific Corporation or its affiliates. All rights reserved.

EP-1985909-AA

CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings, and instructions for use can be found in the product labelling supplied with each device or at [www.IFU-BSCI.com](http://www.IFU-BSCI.com). Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries. This material not intended for use in France.

CE 0459