



VersaCross Radiofrequency System Reduces Time to Left Atrial Access versus Conventional Mechanical Needle

HIGHLIGHTS

The study found LAAC sheath delivery with the **VersaCross™** RF Transseptal Solution was:

- ▶ Efficient: Transseptal puncture and LAAC sheath delivery on average in under 7 mins.
- ▶ Exchangeless: Faster LA access by combining a starter wire, RF transseptal device, and exchange rail in a 3-in-1 solution.
- ▶ Effortless: Controlled RF puncture with a single wire.

INTRODUCTION

- ▶ Left atrial (LA) catheterization requires numerous device exchange steps, and has associated risks and safety concerns.
- ▶ The **VersaCross™** RF Transseptal Solution (Baylis Medical*) enables vascular cannulation, transseptal puncture (TSP), and device exchange using a single RF-tipped pigtail wire.

METHODS

- ▶ Consecutive series of left atrial appendage closure (LAAC) using WATCHMAN™ (Boston Scientific) or Amulet™ (Abbott) devices were retrospectively evaluated.
- ▶ Femoral access was obtained for inferoposterior TSP using two methods:

Conventional approach (n=10):

- Requiring a starter wire, sharp mechanical needle (BRK-1™ Transseptal Needle, Abbott), fixed curve sheath (Swartz™ Transseptal Guiding Introducers, Abbott), and stiff exchange wire (Amplatz Super Stiff™, Boston Scientific or ProTrack™ Pigtail Wire, Baylis Medical)

VersaCross™ RF Transseptal Solution (n=10):

- Comprised of the **VersaCross™** RF Wire, Sheath, and Dilator
- ▶ Efficiency was assessed in terms of time from femoral access to TSP, delivery of LAAC sheath in the LA, device release, overall procedure, and fluoroscopy use.
- ▶ Safety was assessed in terms of intraprocedural and in-hospital complications.

RESULTS

- ▶ LAAC success was 100% using both methods, with no complications.
- ▶ Significant improvement in LA access times using **VersaCross™** RF Transseptal Solution vs. conventional method:
 - Shorter time to TSP [4.1±2.5 min vs. 8.4±4.0 min (p=0.009)]
 - Less time for LAAC delivery sheath into LA [6.7±2.4 min vs. 13.4±5.4 min (p=0.002; Figure 1)]
- ▶ Trend for overall procedural improvement using **VersaCross™** RF Transseptal Solution vs. conventional method:
 - Shorter time to device release [23.7±6.4 min vs. 31.2±10.0 min (p=0.062)]
 - Less fluoroscopy use [7.2±2.2 min vs. 11.4±5.9 min (p=0.061)]

Time for LAAC Sheath Delivery

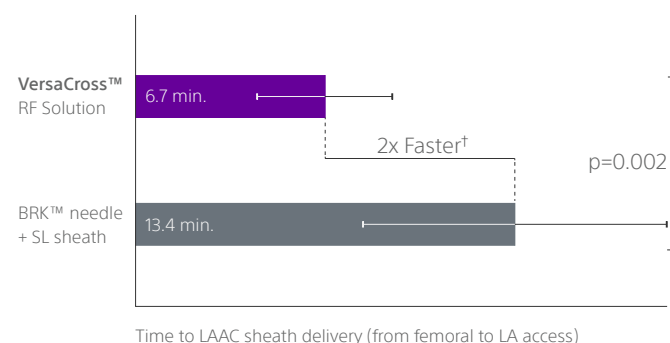


Figure 1. LAAC sheath delivery is two times faster using the **VersaCross™** RF Transseptal Solution than the conventional workflow.†

DISCUSSION & CONCLUSIONS

- ▶ **VersaCross™** RF Transseptal Solution combines a starter wire, transseptal needle, and exchange guidewire for faster LA access, and may improve overall procedural efficiency.

* A wholly-owned subsidiary of Boston Scientific Corporation.

† From femoral access to LAAC sheath delivery using **VersaCross™** RF Transseptal Solution compared to a conventional mechanical needle and fixed curve sheath (Inohara et al., 2021).

All trademarks are property of their respective owners. Patents Pending and/or issued. CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, Contraindications, Warnings, and Instructions For Use can be found in the product labelling supplied with each device or at www.baylismedical.com.

Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries. This material not intended for use in France.



**Boston
Scientific**
Advancing science for life™

© 2023 Boston Scientific Corporation
or its affiliates. All rights reserved.

EP-1548701-AA