



# TheraSphere™ Y-90 Glass Microspheres | DOSISPHERE-01 Trial

**Level I randomised trial showed that unresectable HCC patients who receive a personalised TheraSphere dose using multicompartment dosimetry had a median OS of 26.6 months– a 16-month improvement compared to the control arm.**


Garin E, Tselikas L, Guiu B et al. Personalized versus standard dosimetry approach of selective internal radiation therapy in patients with locally advanced hepatocellular carcinoma (DOSISPHERE-01): a randomised, multicentre, open-label phase 2 trial. *Lancet Gastroenterol Hepatol.* 2021, 6: 17-29

*“Personalised dosimetry is safe and leads to a meaningful improvement in the objective response rate and overall survival of patients with locally intermediate/advanced hepatocellular carcinoma [...] when compared with standard dosimetry.”*

## STUDY OBJECTIVE


A **randomised, multicenter**, investigator sponsored phase II trial comparing the clinical outcomes of SIRT with TheraSphere in patients with intermediate/advanced HCC using two pre-treatment dosimetry determination methods: (1) Standard, single-compartment dosimetry (SDA); defined as a uniform distribution of absorbed dose within the perfused volume – both tumor and normal liver or (2) Personalised (PDA); defined as multi-compartment Y-90 distribution of absorbed dose within the perfused volume that accounts for preferential blood flow into the tumor compared with normal parenchyma.

## STUDY DESIGN

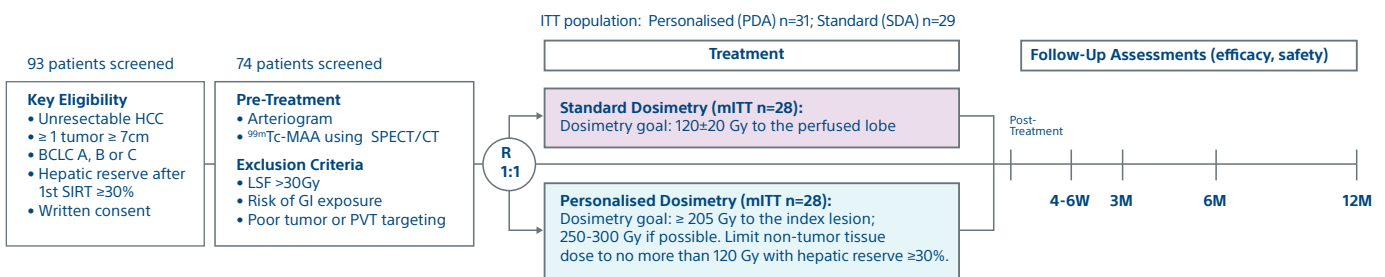


## KEY RESULTS: PERSONALISED DOSIMETRY IMPROVED SURVIVAL

### MEDIAN OVERALL SURVIVAL (ITT POPULATION)

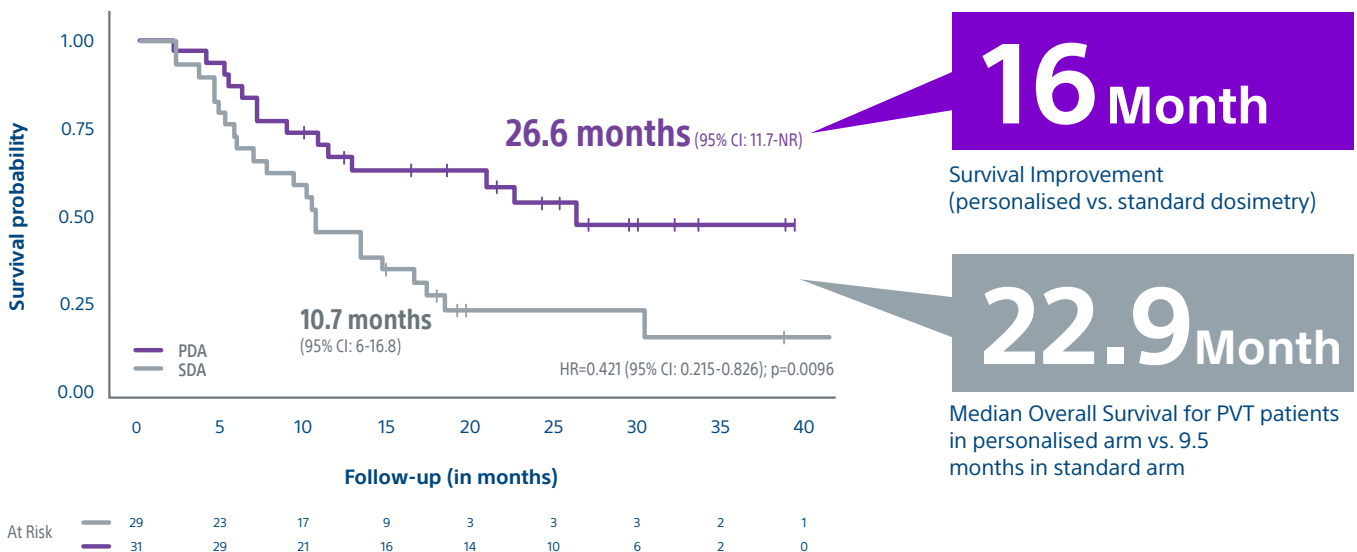

## PERSONALISED DOSIMETRY IMPROVED RESPONSE

### INDEX LESION RESPONSE RATE AT 3 MONTHS USING EASL IN THE MITT POPULATION

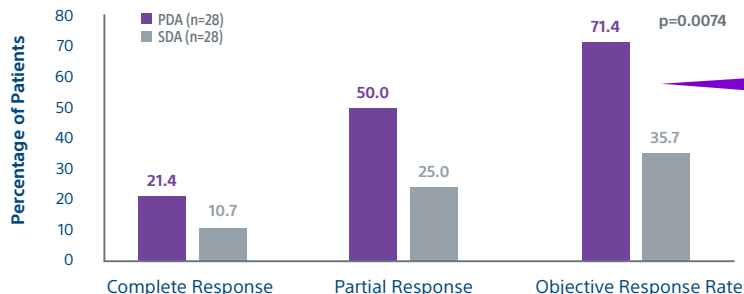


### PRIMARY STUDY ENDPOINT

# 71.4%


Objective Response Rate  
(personalised vs. 35.7%  
in standard dosimetry)

## PERSONALISED DOSIMETRY DOWNSTAGED MORE PATIENTS TO SURGERY

### PATIENTS SUCCESSFULLY DOWNSTAGED TO SURGERY

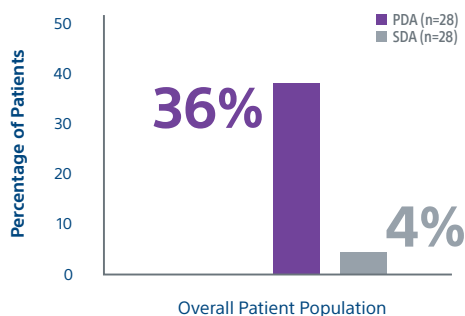

36% of patients in the personalised arm were downstaged vs. 4% in the standardised arm

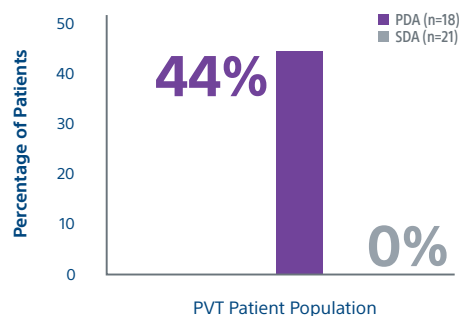

44% of PVT patients in the personalised arm were downstaged vs. 0% in the standardised arm

### DOSISPHERE-01 EDITORIAL:

*"The DOSISPHERE-01 Study challenges the evolving narrative that patients with advanced hepatocellular carcinoma should have systemic therapy at the expense of locoregional therapy. [...] Personalised dosimetry (ie, reaching specific threshold radiation doses) is a natural evolution of selective internal radiation therapy with <sup>90</sup>Y-labelled microspheres."*<sup>3</sup>

– Robert J Lewandowski, MD, Riad Salem, MD, DOSISPHERE Editorial, *Lancet Gastroenterology & Hepatology*

1. Reasons for censoring: received another anti-cancer treatment before M3 evaluation (n=2), no evaluation at M3 evaluation (n=1) (10.7%)  
 2. Reasons for censoring: early deaths (before M3) (n=2), no evaluation at M3 (n=1), start another anti-cancer treatment before M3 evaluation (n=1) (14.3%)  
 3. Lewandowski, Robert J, Salem, Riad. Radioembolisation with personalised dosimetry: improving outcomes for patients with advanced hepatocellular carcinoma. *Lancet Gastroenterol Hepatol* 2020; Published Online: November 06, 2020 [https://doi.org/10.1016/S2468-1253\(20\)30306-X](https://doi.org/10.1016/S2468-1253(20)30306-X)

# THERASPHERE™ Y-90 Glass Microspheres | DOSISPHERE-01 TRIAL

## PATIENT DEMOGRAPHICS (mITT population)

Parameter	PDA (n=28)	SDA (n=28)
Male (%)	92.9	92.9
Child-Pugh Status (%)	CP A5: 78.6 CP A6/B7: 21.4	CP A5: 78.6 CP A6/B7: 21.4
BCLC (%)	BCLC B = 11 BCLC C = 89	BCLC B = 7 BCLC C = 93
Bilobar (%)	43	57
Mean Total Bilirubin (µM/L±SD)	14.0±6.4	14.3±6.4
PVT present (%)	64.3	75.0
PVT Location (%)	Segmental 29.6 Main/Lobar 30/33	Segmental 32.1 Main/Lobar 32/43
Index lesion (mean, cm)	10.5±2.4	10.9±2.57

## TREATMENT CHARACTERISTICS AND DOSIMETRY (mITT population)

Investigator Assessments	PDA (n=28)	SDA (n=28)	P value
Number of Y-90 glass microspheres treatment	One treatment, n=26 Two treatments, n=2	One treatment, n=23 Two treatments, n=5	ns (not significant)
Activity administered GBq (mean, min-max)	3.6 (2.4-4.8)	2.6 (2.2-3.0)	0.0049
AD* to perfused liver (Gy) Mean (±SD)	178.4±59.9	120.3±15.2	0.0001
% of patients with AD to perfused liver > 150 Gy	68	4	<0.0001
AD to index lesion (Gy) Mean (±SD)	331.1±131.5	221.3±139.4	0.0007
% of patients with AD to index lesion > 205 Gy	88	38	<0.0008
AD to perfused normal tissue (Gy) Mean (±SD)	92.8±30.1	64.5±36.6	0.007

\*AD=absorbed dose

## LIVER ADVERSE EVENTS (Grade ≥3) Related to Y-90\*

	PDA (n=35)	SDA (n=21)
Patients with ≥ 1 AE	3 (8.6%)	3 (14.3%)
Death	1 (2.8%)	1 (4.7%)
Liver AEs	4 (11.4%)	5 (23.8%)
Ascites	1	1
Encephalopathy	0	0
GI hemorrhage	0	2
Bilirubin increase/jaundice	1	2
Hepatic failure	2	0

\*patients allocated to either PDA or SDA based on treatment received (dose received) versus allocation by randomisation

Boston Scientific is not responsible for the collection, analysis or reporting of the investigator-sponsored research output which is the sole responsibility of the investigators. Boston Scientific's involvement in investigator-sponsored research is limited to providing financial support for research that advances medical and scientific knowledge about our products.

**CAUTION:** The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings, and instructions for use can be found in the product labelling supplied with each device or at [www.IFU-BSI.com](http://www.IFU-BSI.com). Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries. This material not intended for use in France.

Results from case studies are not necessarily predictive of results in other cases. Results in other cases may vary.

**Note:** Dose to the liver does not exceed 150 Gy. The physician should always take the above-noted Pre-treatment High Risk Factors into consideration for each patient when making decisions regarding the use of TheraSphere for treatment. TheraSphere is a registered trademark of Theragenics Corporation, used under license by Boston Scientific Medical Device Limited, a wholly owned indirect subsidiary of Boston Scientific Corporation.

**Boston  
Scientific**  
Advancing science for life™

[www.bostonscientific.eu](http://www.bostonscientific.eu)

© 2023 Boston Scientific Corporation or its affiliates. All rights reserved.

PI-931202-AB

CE 0123