

REF 3200

PROGRAMMER USER'S MANUAL

EMBLEM™ S-ICD Programmer



CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician trained or experienced in device implant and follow-up procedures.

LIST OF ACRONYMS

AC	Alternating current	LCD	Liquid crystal display
AF	Atrial Fibrillation	MRI	Magnetic resonance imaging
ATP	Anti-tachycardia pacing	NSR	Normal sinus rhythm
CRT	Cardiac resynchronization therapy	RF	Radio Frequency
CPR	Cardiopulmonary resuscitation	RFI	Radio Frequency Interference
ECG	Electrocardiogram	RFID	Radio Frequency Identification
EMI	Electromagnetic interference	S-ECG	Subcutaneous electrocardiogram
EOL	End of life	S-ICD	Subcutaneous implantable cardioverter defibrillator
ERI	Elective replacement indicator	USB	Universal serial bus
ESD	Electrostatic discharge	VAC	Voltage alternating current
FCC	Federal Communications Commission	VF	Ventricular fibrillation
GUI	Graphic User Interface	VT	Ventricular tachycardia
IC	Industry Canada		

This literature is intended for use by professionals trained or experienced in device implant and/or follow-up procedures.

The following are trademarks of Boston Scientific Corporation or its affiliates: EMBLEM, IMAGEREADY, AF Monitor.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks is under license.

microSD™ is a trademark or registered trademark of SD-3C, LLC

This product may be protected by one or more patents.

Patent information can be obtained at <http://www.bostonscientific.com/patents>.

© Copyright 2021 Boston Scientific Corporation or its affiliates. All rights reserved.

TABLE OF CONTENTS

GENERAL DESCRIPTION	1
Description.....	1
Intended Use of Programmer	1
Indications for Use	1
Contraindications.....	1
Related Information.....	2
Programmer Warnings and Precautions.....	2
Programmer Warnings.....	2
<i>General.....</i>	2
<i>Conditions for Operation.....</i>	3
Programmer Precautions	4
<i>General.....</i>	4
<i>Conditions for Operation.....</i>	4
<i>Implantation.....</i>	5
<i>Storage and Handling</i>	5
Potential Adverse Events	5
OPERATION	9
Programmer Setup.....	9
Packaging.....	9
Programmer Controls and Connections	9
Charging the Programmer	10
Using the Programmer.....	11
Turning the Programmer On.....	11
Changing the Programmer Volume Level.....	11
Placing the Programmer in Suspend Mode	11
Turning the Programmer Off	11
Using the Programmer Touch Screen.....	12

Using the Wand	12
Navigation.....	13
Screen Header.....	13
Navigation Bar	14
Restarting the Programmer	15
Configuring the Programmer.....	16
Configuring Programmer Settings.....	16
Date and Time Format	17
Time Zone	18
Language Preference	19
Printer Selection.....	19
Programmer Software Version.....	20
Bluetooth® Data Export	21
Programmer Modes of Operation.....	22
Online Behavior	22
Offline Behavior	23
Stored Patient Sessions.....	23
To view stored patient sessions:.....	23
To purge stored patient sessions:.....	23
Modes of Operation for the Pulse Generator	23
Shelf Mode	24
Therapy On Mode	24
Therapy Off Mode	24
MRI Protection Mode	24
Connecting and Disconnecting from the S-ICD Pulse Generator	25
Scanning for Pulse Generators.....	25
Connecting to a Pulse Generator	26
Connecting to a Pulse Generator in Shelf Mode	26

Connecting to an Implanted Pulse Generator	27
Ending a Patient Session.....	27
Programming the Pulse Generator at Implant	29
Entering Electrode Information.....	30
Creating the Patient Chart.....	31
Automatic Setup	32
Programming Therapy Parameters.....	34
Defibrillation Testing	36
Performing a Follow Up	39
Printing Reports from the Programmer.....	40
Printing Reports.....	40
Summary Report.....	41
Captured S-ECG Report	42
Episode Reports.....	42
Export Patient Data	43
Export using <i>Bluetooth</i> [®] wireless technology	43
Export using a microSD [™] card.....	44
S-ECG Features.....	45
S-ECG Rhythm Strip Markers.....	45
S-ECG Scale Settings	46
Capturing and viewing S-ECG Strips	47
To manually capture a new S-ECG rhythm strip:	48
Viewing previously captured S-ECGs	48
Utilities Screen	49
Acquire Reference S-ECG.....	49
Capture All Sense Vectors	50
Beeper Control	50
Reset Beeper	51

<i>Disable Beeper (SQ-Rx devices)</i>	51
<i>Enable/Disable Beeper (EMBLEM S-ICD devices)</i>	52
Manual Setup.....	52
SMART Settings.....	54
<i>SMART Charge</i>	54
<i>Disabling SMART Pass</i>	55
AF Monitor	55
Additional Programmer Functions	56
Rescue Shock	56
Manual Shock.....	58
S-ICD System Magnet Use.....	58
MAINTENANCE	59
Charging the Programmer	59
Cleaning the Programmer and Wand	59
Service.....	60
Maintenance Check.....	60
Safety Measurements	60
Disposal of Programmer and Accessories.....	61
TROUBLESHOOTING.....	61
Inability to Print	61
No Printer Available	61
Touch Screen Inactive while Connected to AC Power.....	62
Loss of Communication with Printer	62
Inability to Communicate With the Pulse Generator.....	62
COMPLIANCE STATEMENTS	62
EMI/RFI	62
Essential Performance	63
Federal Communications Commission (FCC) Compliance.....	63

<i>Industry Canada (IC) Compliance</i>	64
DECLARATIONS TABLES	64
SPECIFICATIONS	67
DEFINITION OF PACKAGING LABEL AND DEVICE SYMBOLS	69
WARRANTY	71
Limited Warranty	71
APPENDIX A: INSERTION AND REMOVAL OF THE microSD™ CARD	72

GENERAL DESCRIPTION

Description

The EMBLEM S-ICD Programmer (the “Programmer”) is a component of Boston Scientific’s subcutaneous implantable cardioverter defibrillator system (the S-ICD System) which is prescribed for patients when cardiac arrhythmia management is warranted. Implantable components of the S-ICD System include the EMBLEM S-ICD pulse generator and the EMBLEM S-ICD subcutaneous electrode.

The Programmer is a non-sterile, non-implantable, tablet computer controlled by a graphic user interface (GUI) displayed on a touchscreen. The Programmer is powered by either AC line power or an internal lithium ion battery pack. The Programmer uses a connected RF telemetry wand to communicate wirelessly with the S-ICD pulse generator in order to adjust programmable settings and to collect patient data. The EMBLEM S-ICD Programmer is also compatible with the Cameron Health (Model 1010) SQ-Rx pulse generator. The Programmer features and functions described in this manual apply to the Boston Scientific S-ICD System as well as to the Cameron Health S-ICD System.

The S-ICD System is designed for ease of use and simplicity of patient management. The S-ICD System has a number of automatic functions designed to reduce the amount of time required for implantation, initial programming and patient follow up.

Go to www.bostonscientific-elabeling.com for additional technical reference guides.

Intended Use of Programmer

The Programmer is intended to communicate with the implanted pulse generator using wireless telemetry. The Programmer software controls all such telemetry functions.

Indications for Use

The S-ICD System is intended to provide defibrillation therapy for the treatment of life-threatening ventricular tachyarrhythmias in patients who do not have symptomatic bradycardia, incessant ventricular tachycardia, or spontaneous, frequently recurring ventricular tachycardia that is reliably terminated with anti-tachycardia pacing.

Contraindications

Unipolar stimulation and impedance-based features are contraindicated for use with the S-ICD System.

Related Information

Before using the S-ICD system, read and follow all instructions, warnings, and precautions provided in this manual and in the manuals for the other system components, including the applicable S-ICD pulse generator, subcutaneous electrode, and electrode implant tools user's manuals.

This guide may contain reference information for pulse generator model numbers that are not currently approved for sale in all geographies. For a complete list of model numbers approved in your geography, consult with your local sales representative. Some model numbers may contain fewer features; for those devices, disregard descriptions of the unavailable features. Descriptions found within this manual apply to all device tiers unless otherwise noted.

Refer to the IMAGEREADY MR Conditional S-ICD System MRI Technical Guide (hereafter referred to as the MRI Technical Guide) for information about MRI scanning.

Programmer Warnings and Precautions

The following warnings and precautions apply specifically to the Model 3200 Programmer component of the S-ICD System.

For Warnings and Precautions that are specific to the S-ICD system and/or the process of implanting the system, refer to the EMBLEM™ S-ICD User's Manual, and/or the manual of the relevant system component.

Programmer Warnings

General

- **Modifications.** No modification of this equipment is allowed unless approved by Boston Scientific.
- **Programmer is MR Unsafe.** The Programmer is MR Unsafe and must remain outside the MRI site Zone III (and higher) as defined by the American College of Radiology Guidance Document on MR Safe Practices¹. Under no circumstances should the Programmer be brought into the MRI scanner room, the control room, or the MRI site Zone III or IV areas.
- **High temperatures.** Do not subject the Programmer to temperatures outside of the -10° C to 55° C (14° F to 131° F) storage range. Exposure to high temperatures may cause the Programmer to overheat or ignite, and may possibly reduce its performance and service life.
- **Extreme temperatures.** Do not discard the Programmer in a fire, incinerate, or subject it to temperatures that exceed 100° C (212° F). This could cause the Programmer to explode.

¹ Kanal E, Barkovich AJ, Bell C, et al. ACR guidance document on MR safe practices: 2013. J. Magn. Reson. Imaging 2013;37:501-530.

- **Do not immerse.** Do not immerse the Programmer in liquid of any kind. If the Programmer does get wet, contact customer service for information about returning the Programmer to Boston Scientific. Do not attempt to dry the Programmer in an oven, microwave, or dryer because this poses a risk of overheating or explosion.

Conditions for Operation

- **Device Software Update.** During a device software update, tachycardia therapy is suspended. Always monitor the patient and have external defibrillation equipment available during interrogation.
- **Secure Programmer.** Ensure this Programmer is used by professionals trained or experienced in device implant and/or follow-up procedures. Take appropriate measures to prevent unauthorized use or tampering of the Programmer.
- **Use only the supplied external power supply.** Use the Programmer only with the external power supply packaged with the Programmer. Using other power supplies may cause damage to the Programmer.
- **Electric shock.** To avoid risk of electric shock, the Programmer's external power supply must only be connected to a grounded electrical outlet.
- **Damaged Programmer or power supply.** Never use a damaged external power supply or a damaged Programmer. Doing so could result in user injury, patient injury, or a lack of therapy delivery.
- **Interference with nearby equipment.** By design, the Programmer emits radio frequencies in the 402-405 MHz and 2.4 GHz bands. This may interfere with nearby medical or office equipment. When using the Programmer, closely monitor equipment in the vicinity to verify normal operation. It may be necessary to take mitigation measures, such as reorienting or relocating the Programmer or shielding the location.
- **Interference with Programmer communication.** The presence of other equipment operating in the same frequency bands used by the Programmer (402-405 MHz for the pulse generator and 2.4 GHz for the printer) may interfere with communication. Interference can occur even if the other equipment complies with the International Special Committee on Radio Interference (CISPR) emission requirements. This RF interference can be reduced by increasing the distance between the interfering device and the Programmer and pulse generator or printer. If communication problems persist, refer to the Troubleshooting section of this manual.
- **Use of non-approved accessories.** The use of any accessories with the Programmer other than those specified by Boston Scientific in this manual may result in increased emissions or decreased immunity of the Programmer and may cause decreased functionality or unintended operational behavior of the Programmer. Anyone connecting such accessories to the Programmer may be configuring a medical system and is responsible to ensure that the system complies with the requirements of IEC/EN 60601-1, Clause 16 for medical electrical systems.

- **Programmer location.** Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- **Radio frequency (RF) communications equipment.** Keep all RF communications equipment (including peripherals such as antennas, wands, and cables) at least 30 cm (12 in) away from the Model 3200 Programmer, including cables specified by Boston Scientific, to avoid degradation of the performance of this equipment.

Programmer Precautions

General

- **Wand use.** Use only the Model 3203 telemetry wand with the Programmer.
- **Do not disassemble.** Do not disassemble or alter any parts of the Programmer.
- **Changes or modifications.** Changes or modifications not expressly approved by Boston Scientific could void the user's authority to operate the equipment.
- **Device communication.** Use only the designated Boston Scientific S-ICD Programmer and appropriate software application to communicate with and program the S-ICD pulse generator.
- **Intended users.** The Programmer is intended for use by or under the direction of healthcare professionals only.
- **Sensitive Information.** To prevent sensitive personal information from being transmitted to inappropriate devices or printers when using Bluetooth® wireless connections, make certain to only connect with known Bluetooth® devices.

Conditions for Operation

- **Beeper Disable.** In the Programmer steps to program and test the Beeper, selecting "No, Disable Beeper" will disable beeping functionality for alert conditions, when a magnet is placed over the device, and when the S-ICD application connects to the device.
- **Disconnecting the Programmer.** Mains isolation is achieved by disconnecting the external power supply power cord from the AC electrical outlet. Do not position the Programmer or the external power supply in a manner that would make it difficult to disconnect that cord.
- **Programmer use.** The Programmer is not waterproof or explosion-proof and cannot be sterilized. Do not use it in the presence of flammable gas mixtures containing anesthetics, oxygen, or nitrous oxide.
- **Confirm communication.** Confirm that the Programmer is in communication with the intended implanted S-ICD pulse generator.

- **Electrostatic discharge.** The Programmer may be affected by ESD. If ESD occurs and the Programmer's functionality is affected, attempt to reset the Programmer or contact Boston Scientific for instructions. Do not touch or connect the telemetry wand to the Programmer unless ESD precautionary procedures are used.

Implantation

- **Telemetry wand.** The wand is a non-sterile device. Do not sterilize the wand. The wand must be contained in a sterile barrier before use in the sterile field.
- **Programmer must remain outside the sterile field.** The Programmer is non-sterile and cannot be sterilized. It must remain outside the sterile field.

Storage and Handling

- **Mishandling.** Mishandling (such as dropping or crushing) could damage the Programmer. If you suspect damage to the Programmer, contact your Boston Scientific representative or the customer service department for instructions and return packaging.
- **Broken or cracked screen.** The display on the Programmer is made of glass or acrylic and could break if the Programmer is dropped or if it receives significant impact. Do not use if screen is broken or cracked as this could cause injury.
- **Magnet handling.** Do not place a magnet on the Programmer.
- **Data Storage.** The Programmer and digital data storage media, such as microSD™ memory cards, used with the Programmer may contain sensitive personal information. These should be handled in accordance with applicable privacy and security policies and regulations.

Potential Adverse Events

Potential adverse events related to implantation of the S-ICD System may include, but are not limited to, the following:

- Acceleration/induction of atrial or ventricular arrhythmia
- Adverse reaction to induction testing
- Allergic/adverse reaction to system or medication
- Bleeding
- Conductor fracture
- Cyst formation
- Death

- Delayed therapy delivery
- Discomfort or prolonged healing of incision
- Electrode deformation and/or breakage
- Electrode insulation failure
- Erosion/extrusion
- Failure to deliver therapy
- Fever
- Hematoma/seroma
- Hemothorax
- Improper electrode connection to the device
- Inability to communicate with the device
- Inability to defibrillate or pace
- Inappropriate post-shock pacing
- Inappropriate shock delivery
- Infection
- Injury to or pain in upper extremity including clavicle, shoulder, and arm
- Keloid formation
- Migration or dislodgement
- Muscle/nerve stimulation
- Nerve damage
- Organ injury or perforation
- Pneumothorax
- Post-shock/post-pace discomfort
- Premature battery depletion
- Random component failures
- Stroke
- Subcutaneous emphysema

- Surgical revision or replacement of the system
- Syncope
- Tissue damage
- Tissue redness, irritation, numbness or necrosis
- Vessel injury or perforation

Transient procedural adverse events are expected in some patients. These include, but are not limited to, discomfort, pain and other systemic symptoms that might be related to medications or other interventions performed during implant.

For a list of potential adverse events associated with MRI scanning, refer to the MRI Technical Guide.

If any adverse events occur, invasive corrective action and/or S-ICD System modification or removal may be required.

Patients who receive an S-ICD System may also develop psychological disorders that include, but are not limited to, the following:

- Depression/anxiety
- Fear of device malfunction
- Fear of shocks
- Phantom shocks

Any serious incident that occurs in relation to this device should be reported to Boston Scientific and relevant local regulatory authority.

OPERATION

Programmer Setup

Packaging

Programmer components include:

- Model 3200 Programmer with pre-loaded software
- Model 3203 Telemetry Wand
- Model 3204 External power supply and AC power cord

Visually inspect the packaging to ensure the contents are complete. Do not use if there is evidence of damage.

In case of damage return the product to Boston Scientific. For return packaging and instructions, contact Boston Scientific using the information on the back cover of this manual.

Programmer Controls and Connections

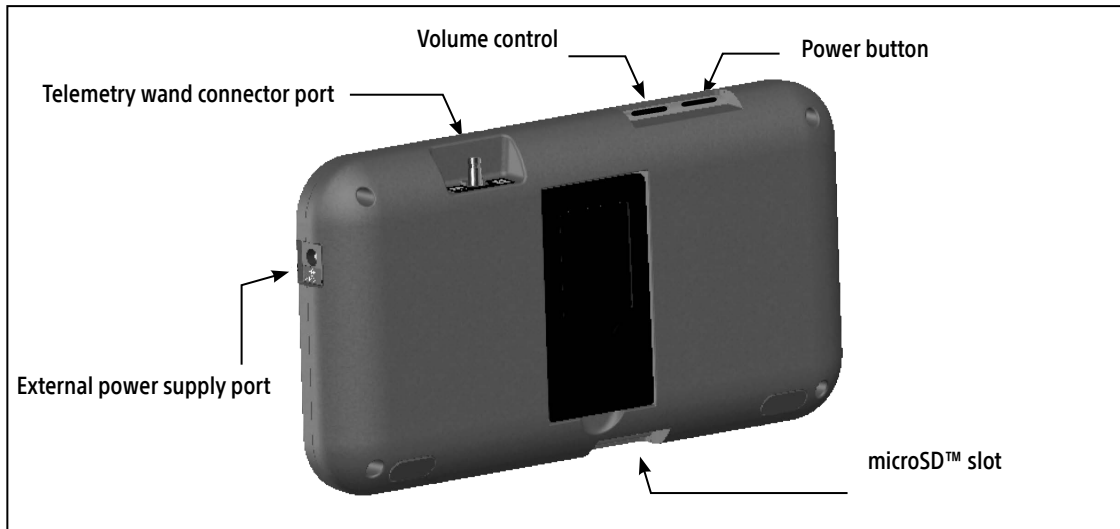


Figure 1: Controls and external connections

Charging the Programmer

The Programmer is primarily intended to be operated while connected to the AC-powered external power supply, but may also be operated on battery power provided that the internal battery is adequately charged. The Programmer is recharged whenever it is connected to the AC-powered external power supply. When not in use, it is recommended that the Programmer remain connected to the external power supply in order to maintain an adequate battery charge.

Note: *Current session data may be lost if a 45 minute period of inactivity occurs during an active telemetry session and the Programmer is not connected to AC power.*

Typical charge time for a fully discharged battery is 5 hours. However, more time may be required if the Programmer is in use while being recharged.

The Battery Status indicator located on the upper right corner of the screen displays the status of the main battery power when the unit is in use:

- All four bars are illuminated (green) – The battery is 100% charged
- Three bars are illuminated (green) – The battery is 75% charged
- Two bars are illuminated (yellow) – The battery is 50% charged
- One bar is illuminated (red) – The battery is 25% charged

The Programmer displays one of the following alert screens as battery power gets progressively lower.

- Programmer Battery Low
- Programmer Battery Critical
- Out Of Power

To charge the Programmer:

1. Connect the external power supply cable to the Programmer (Figure 1 on page 9).
2. Plug the external power supply cord into an AC power outlet.

WARNING: Use the Programmer only with the external power supply packaged with the Programmer. Using other power supplies may cause damage to the Programmer.

WARNING: To avoid risk of electric shock, the Programmer's external power supply must only be connected to a grounded electrical outlet.

Using the Programmer

WARNING: During a device software update, tachycardia therapy is suspended. Always monitor the patient and have external defibrillation equipment available during interrogation.

Turning the Programmer On

The Programmer power button is located in the recess above and behind the left corner of the screen (Figure 1). Press and hold the button until the display screen is active.

Note: *If the Programmer cannot be turned on while it is connected to AC power via the external power supply, first unplug the external power supply cord from the Programmer. Press and hold the Programmer power button until the display screen is active. AC power via the external power supply can then be reconnected.*

Changing the Programmer Volume Level

The volume level of Programmer-generated sounds may be temporarily adjusted using the volume control (Figure 1 on page 9). This level is automatically reset when the Programmer is restarted.

Placing the Programmer in Suspend Mode

The Programmer has a Suspend Mode which is activated automatically to conserve power. The display will be blank when this mode is in effect.

The Programmer enters Suspend Mode whenever:

- The power button is momentarily pressed and released
- The Programmer is not connected to the external power supply, it is not in active communication with an S-ICD pulse generator, and no user activity has occurred for 15 minutes

Momentarily pressing the power button will resume normal operation.

Turning the Programmer Off

There are two ways to turn the Programmer off:

1. Press and hold the power button until the System shutdown menu appears. Select Power off from the popup and confirm by pressing OK.

- From the Programmer start-up screen, press the Power Off button and select OK at the confirmation prompt.

Using the Programmer Touch Screen

The Programmer is equipped with an LCD touch screen. The screen can be adjusted to the desired viewing angle by using the kick-stand located on the back of the Programmer. All interaction with the Programmer is conducted using the fingers to touch the appropriate areas on the screen. Scroll on-screen lists by sliding a finger up and down the list. An on-screen keyboard is presented whenever text entry is required.

Caution: The display on the Programmer is made of glass or acrylic and could break if the Programmer is dropped or if it receives significant impact. Do not use if screen is broken or cracked as this could cause injury.

Using the Wand

The Model 3203 wand (“the wand”) makes it possible for this Programmer to communicate with the pulse generator.

Caution: Use only the Model 3203 telemetry wand with the Programmer.

Caution: The wand is a non-sterile device. Do not sterilize the wand. The wand must be contained in a sterile barrier before use in the sterile field

Caution: The Programmer is non-sterile and cannot be sterilized. It must remain outside the sterile field.

To connect the wand to the Programmer, slide the wand cable connector over the communication connector port located on the rear edge of the Programmer (Figure 1 on page 9).

To disconnect the wand, grasp the wand cable connector and gently pull it straight off the communication connector port.

Note: *Do not pull or yank on the cable to disconnect the wand from the Programmer. Such action could cause hidden damage to the cable. A damaged cable might reduce wireless communication capabilities and require a replacement wand.*

Optimal telemetry depends on the wand being placed directly over the implanted pulse generator. Although it may appear that the Programmer is in communication with the pulse generator at greater distances, programming should always be performed with the wand placed directly over the implanted pulse generator.

WARNING: The presence of other equipment operating in the same frequency bands used by the Programmer (402-405 MHz for the pulse generator and 2.4 GHz for the printer) may interfere with communication. Interference can occur even if the other equipment complies with the International Special Committee on Radio Interference (CISPR) emission requirements. This RF interference can be reduced by increasing the distance between the interfering device and the Programmer and pulse generator or printer. If communication problems persist, refer to the Troubleshooting section of this manual.

When telemetry loss occurs, the display screen will turn yellow and a message will appear with the text “Communication Loss” to alert the user. Reposition the wand to establish communication. The Programmer will return to the screen that was active before telemetry loss if the pulse generator is found and programming can continue.

Note: *If communication cannot be reestablished, the session should be ended and restarted by scanning for the pulse generator.*

Navigation

The Programmer’s graphic user interface (GUI) facilitates management and control of the S-ICD System. The Navigation Bar and on-screen icons at the top of the screen allow the user to navigate programming software screens. In addition, a continuous subcutaneous electrocardiogram (S-ECG) is displayed along the bottom of the screen during Online (active) communication with the pulse generator.

Screen Header

When the Programmer is Offline (inactive communication), the screen header displays the Battery Status Indicator.

When viewing Offline Stored Sessions, the screen header displays:

- Patient name
- Therapy On/Off
- Battery status indicator

When the Programmer is Online (active communication), the screen header displays:

- Therapy On/Off
- Patient name
- Patient heart rate

- Programmer Battery and Telemetry status indicator
- Screen title
- Rescue shock icon

Navigation Bar

The Navigation Bar is the primary method for navigating the Online Programmer screens. The bar is located along the top edge of the Programmer screen and chosen screens appear with their selection icon highlighted.

Table 1: Icon descriptions on page 14 provides a list of the Programmer icons and their corresponding descriptions.

Table 1: Icon descriptions















Icon	Description	User Application
	Main Menu Icon	Allows user to return to the main menu.
	Automatic Setup Icon	Allows user to access the Automatic Setup menu.
	Device Settings Icon	Allows user to access the S-ICD device settings screen.
	Device Status Icon (open folder and closed folder)	Allows user to access the S-ICD device status screen. User can view number of shocks delivered since the last update as well as the S-ICD device battery life.
	Patient View Icon	Allows user to access the patient chart screen. User can view information on the S-ICD device battery life.
	Captured and Stored Episodes S-ECG Icon	Allows user to access captured S-ECG and stored episode screens.
	Induction Test Icon	Allows user to access induction screen.

Table 1: Icon descriptions

Icon	Description	User Application
	Manual Shock Icon	Allows user to access the manual shock screen.
	Battery & Telemetry Meter	Left side of the meter allows user to view the Programmer's battery status. The right side of the meter allows viewing of telemetry signal strength.
	Capture S-ECG	Allows user to capture a live S-ECG.
	S-ECG Display Settings	Allows user to modify the zoom and sweep speed on the live S-ECG.
	Heart Rate Icon	Allows user to view current heart rate.
	Rescue Shock Icon	Allows user to administer a rescue shock
	Option Selection Switch	Allows user to select one of two options, e.g. A or B

Restarting the Programmer

The Programmer's operating system is self-monitoring and is generally able to sense many system error conditions and automatically initiate a restart sequence in response. Follow the on-screen instructions to complete the Programmer-initiated restart sequence.

The Programmer may need to be manually restarted if:

- You cannot exit a screen
- The operating system stops responding

A manual restart is accomplished by pressing and holding the power button until the system shutdown menu appears on the screen. Select Restart from the popup and confirm by pressing OK.

If the Programmer does not respond to a restart process, contact Boston Scientific using the information on the back cover of this manual.

Configuring the Programmer

Configuring Programmer Settings

The Programmer should be configured before communication with a pulse generator is attempted. This includes setting the date and time format, time zone, language and printer. Once these settings are configured during the initial setup process they become the default parameters and will not normally need to be changed with each session.

To configure the Programmer settings:

1. Select the Programmer Settings button on the Programmer start-up screen (Figure 2) to display the Programmer Settings screen (Figure 3 on page 17).

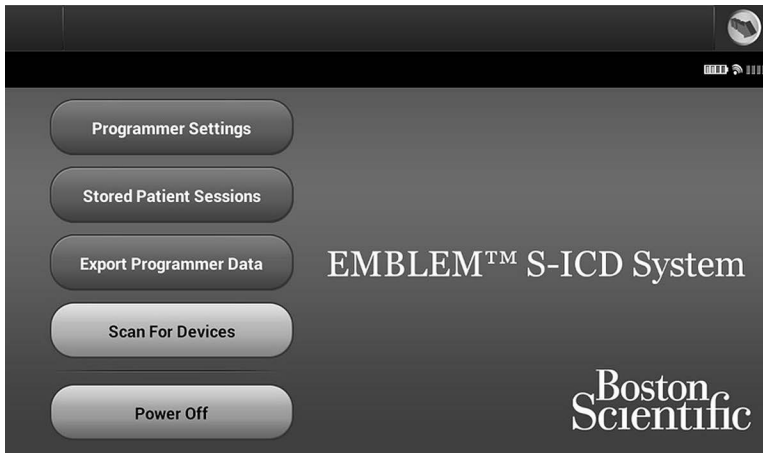


Figure 2: Programmer start-up screen

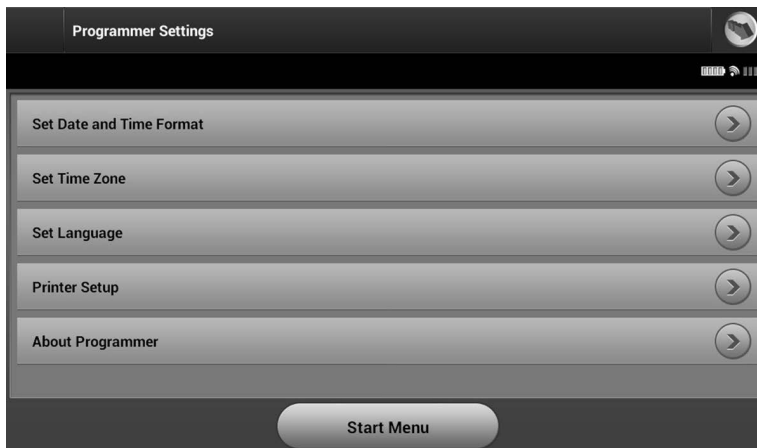


Figure 3: Programmer Settings screen

2. Select the corresponding line to access each setting. The settings that can be configured include:
 - Date and time format
 - Time zone
 - Language
 - Printer

Date and Time Format

To set the date and time format:

1. Select Set Date and Time Format on the Programmer Settings screen (Figure 3). The Date and Time Settings screen appears.
2. Select the desired date format.
3. Select the Save button to save the changes and return to the Programmer Settings screen, or select Cancel to return to the Programmer Settings screen without saving the changes.

Time Zone

The time zone setting controls two S-ICD System parameters, one for the Programmer (the time shown on screens and printed reports), the other for pulse generators (the electronic filter that is intended to minimize electromagnetic interference {EMI}).

Choosing the correct time zone setting for the Programmer will result in the electronic filter of interrogated pulse generators being set to the appropriate regional electrical power line frequency.

Specifically, the pulse generator line frequency filter is automatically programmed to either 50 Hz or 60 Hz, based on the time zone setting of the interrogating Programmer.

To set the time zone

1. Select Set Time Zone on the Programmer Settings screen. The time zone selection screen appears (Figure 4 on page 19).
2. Select the time zone button for the zone in which the Programmer will be used. A check mark will appear in the selected button.
3. Select the Save button to save the changes and return to the Programmer Settings screen, or select Cancel to return to the Programmer Settings screen without saving the changes.

In the rare cases where a single time zone setting includes regional power line frequency differences, two line frequency options are available. Choose the option with the correct frequency for the region where the Programmer is located.

Because a Programmer will set the time zone (and electronic frequency filter) of pulse generators it interrogates to match its own time zone setting, be aware that travelling patients whose devices are interrogated in time zones or countries other than the one in which they reside may need to have their pulse generator time zone reset upon returning home.

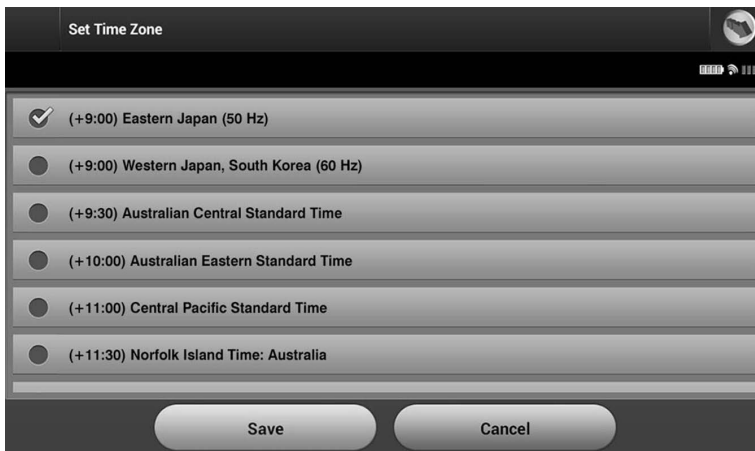


Figure 4: Set Time Zone screen (scrollable list)

Language Preference

To set the language preference:

1. Select Set Language on the Programmer Settings screen. The Language Settings screen appears. Scroll the list and select a language.
2. Select the Save button to save the changes or select Cancel to return to the Programmer Settings screen without saving the changes. If the language is changed the Programmer will automatically restart and return to the Startup screen.

Printer Selection

The Programmer communicates with the printer via *Bluetooth*® wireless technology. Only Boston Scientific-approved printers should be paired and used with the Programmer. To select a printer to be paired and used with the Programmer:

Note: *Some printers require pairing confirmation on the Programmer and printer. In such a case, refer to the printer manufacturer's documentation for more detail.*

1. Ensure the printer is on and, depending on your specific printer, that the wireless function is enabled or the wireless adapter is in the printer's USB port.

2. Select Printer Setup on the Programmer Settings screen. The Printer Setup screen (Figure 5) may appear with a previously configured printer displayed as the default printer. If a default printer has not already been selected and configured, the screen will be empty and the Programmer will scan the area to locate wireless printers. A Scan Progress Bar will appear informing the user that the Programmer is currently scanning for printers.

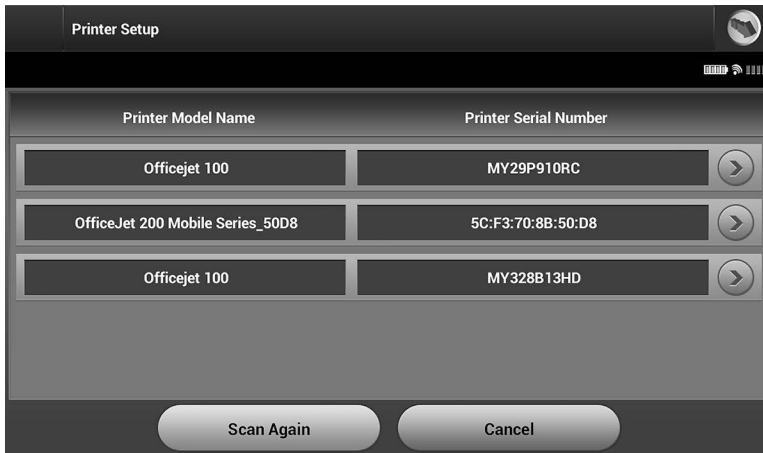


Figure 5: Printer Setup screen

3. Select the printer of choice from among those found during the scan. If none were found, a window will appear stating that there are no printers. Rescan or select the Cancel button to return to the Programmer Settings screen.
4. Optionally, select the desired printer from the list and rename it using the on-screen keyboard (up to 15 characters). The printer serial number will appear with the printer selection.
5. Select the Save button to save the changes and return to the Programmer Settings screen, or select Cancel to return to the Programmer Settings screen without saving the changes. A confirmation screen will appear when the printer setup is completed.

Note: Refer to the Troubleshooting section for information about printer problems.

Programmer Software Version

To view the Programmer's software version:

1. Select About Programmer on the Programmer Settings screen. The Programmer Software Version information screen appears.
2. The Programmer Software Version information screen displays the current version of the Programmer software. Select the Continue button to return to the Programmer Settings screen.

Note: *The printed reports also contain the Programmer software versions.*

Bluetooth® Data Export

The Programmer can be configured to wirelessly export patient data to desktop or notebook computers that are equipped with *Bluetooth®* wireless technology. The Programmer and each computer must be individually paired in order to use the wireless data export function. The procedure for pairing the Programmer with a computer is different from the procedure used to pair the Programmer with the printer.

Note: *Data transfer is supported for computers running Windows. The data transfer feature is not available for tablets or smartphones.*

1. Ensure the computer to be paired has been made discoverable, since the Programmer searches for nearby computers during the pairing process.

Note: *Detailed instructions for accomplishing this are found in the Microsoft Windows help files under the general heading of “Why can’t I connect my Bluetooth® device to my computer?”*

2. Once the target computer has been made discoverable, select the Export Programmer Data button on the Programmer start-up screen. The Export Programmer Data Over *Bluetooth®* screen will appear. Select the Set Up Authorized Computers button to scan for nearby computers and begin the pairing process.
3. When the scan is complete the screen will list the discovered computers (the three computers having the strongest *Bluetooth®* signals) under the heading Unauthorized Computers Nearby (Figure 6). Choose the computer you wish to pair with and press the plus button next to it to complete the pairing process.
4. During the pairing process, both the Programmer and the computer will present identical numeric passkeys and both machines will ask you to confirm that the two numbers are the same. The passkey is only presented while pairing and is used to verify that the correct machines are being paired.
5. Successful pairing is indicated when the listing for the subject computer appears in the Authorized Computers column instead of the Unauthorized Computers Nearby column.
6. Authorized computers may be renamed if desired. Press and hold the listing for the computer until the Rename an Authorized Computer popup appears.

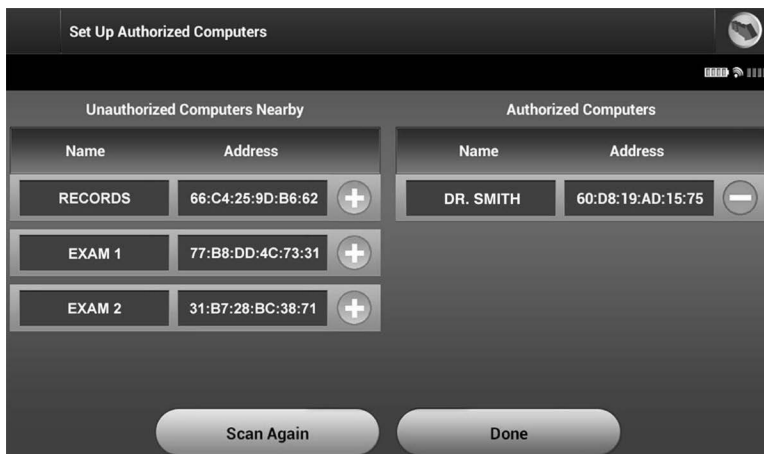


Figure 6: Choosing a computer to authorize for *Bluetooth*® data transfer

Programmer Modes of Operation

Online Behavior

The Programmer's interface varies according to whether the Programmer is Online (actively communicating) or Offline (not communicating) with a selected pulse generator.

An Online session begins when the Programmer establishes a telemetry link with a specific pulse generator. A yellow alert screen is displayed if the telemetry signal is lost between the Programmer and the pulse generator for more than five seconds during active communication. This may occur if the wand is moved out of the telemetry communications range or if noise or interfering objects inhibit communication. Programming commands, including Rescue Shocks, will not be available until telemetry is reestablished.

Telemetry reconnection may occur automatically if the reason for the telemetry loss has been remedied, e.g. moving the wand back into telemetry range of the pulse generator or removing the source of interference or noise. Restart the session if the telemetry link does not resume within one minute.

Note: *When in active communication with a pulse generator, the Programmer emits an audible notification to indicate that the pulse generator is preparing to deliver a shock, whether that shock is commanded or is in response to a detected arrhythmia. The notification continues until the shock is either delivered or aborted.*

Offline Behavior

The Programmer is Offline when it is not actively communicating with a pulse generator. Programmer settings can be accessed and stored patient sessions can be viewed and/or printed during Offline sessions.

Stored Patient Sessions

During a patient follow-up visit, the Programmer will retrieve data from the pulse generator memory. The Programmer will store up to 50 patient sessions or 90 days of patient session data. When the 51st session occurs, the Programmer will automatically replace the oldest stored session with the new data. Additionally, when the 90th day passes after a patient session was stored, the Programmer will automatically delete that patient session when the Programmer is next powered up.

A stored session includes the following information:

- **Captured S-ECG Reports (including Induction S-ECGs)**
- **Episode History (including any downloaded episodes)**
- **Patient Data**
- **Programmed Device Settings**

To view stored patient sessions:

1. From the Programmer start-up screen, select Stored Patient Sessions.
2. Select the desired patient session.

To purge stored patient sessions:

1. From the Programmer start-up screen, select Stored Patient Sessions.
2. Select the Purge All Data button.
3. A confirmation dialog appears. Select OK to purge all the stored patient session data.

Modes of Operation for the Pulse Generator

The pulse generator has the following modes of operation:

- Shelf
- Therapy On
- Therapy Off

- MRI Protection Mode

Shelf Mode

The Shelf mode is a low power consumption state intended for storage only. When a pulse generator in Shelf mode is interrogated by a Programmer, it exits Shelf mode and defaults to Therapy Off mode. A full-energy capacitor reformation is performed and the pulse generator is prepared for set-up. Once the pulse generator is taken out of Shelf mode, it cannot be reprogrammed back into Shelf mode.

Therapy On Mode

The Therapy On mode is the primary operating mode of the pulse generator, allowing automatic detection of, and response to, ventricular tachyarrhythmias.

Therapy Off Mode

The Therapy Off mode disables automatic therapy delivery while still allowing manual control of shock delivery. Programmable parameters may be viewed and adjusted via the Programmer. The subcutaneous electrogram (S-ECG) may be displayed or printed from this mode.

The pulse generator defaults to Therapy Off mode when it is taken out of Shelf mode.

Note: *Manual and rescue shock therapy are available when the device is set to Therapy On or Therapy Off mode and is actively communicating with a pulse generator, but only after the initial Setup process is complete. Refer to Automatic Setup on page 32.*

MRI Protection Mode

MRI Protection Mode is available in EMBLEM S-ICD devices.

MRI Protection Mode modifies certain pulse generator functions in order to mitigate risks associated with exposing the S-ICD system to the MRI environment. Choosing MRI Protection Mode will initiate a sequence of screens to assess the patient's eligibility and readiness to undergo an MR Conditional MRI scan. Refer to the Summary Report to find out whether the device has been in MRI Protection Mode. For a complete description of MRI Protection Mode, a list of MR Conditional devices, and additional information about the IMAGEREADY S-ICD System, refer to the MRI Technical Guide.

Prior to the patient undergoing an MRI scan, an IMAGEREADY S-ICD System must be programmed to the MRI Protection Mode using the Programmer. In MRI Protection Mode:

- Tachycardia therapy is suspended
- A Time-out feature is nominally set to 6 hours, with programmable values of 6, 9, 12, and 24 hours
- Beeper is disabled

MRI Protection Mode is terminated by manual exit or through the user-programmed automatic MRI Protection Time-out period (refer to the MRI Technical Guide for MRI Protection Mode programming instructions.) Rescue Shock will also terminate MRI Protection Mode. When MRI Protection Mode is exited, all parameters (except for the beeper) return to the previously programmed settings.

WARNING: The beeper may no longer be usable following an MRI scan. Coming in contact with the strong magnetic field of an MRI scanner may cause a permanent loss of the beeper volume. This cannot be recovered, even after leaving the MR scan environment and exiting MRI Protection Mode. Before an MRI procedure is performed, a physician and patient should weigh the benefit of the MR procedure against the risk of losing the beeper. It is strongly recommended that patients are followed on LATITUDE NXT after an MRI scan if they are not already. Otherwise, an in-clinic follow-up schedule of every three months is strongly recommended to monitor device performance.

Note: *The beeper can be re-enabled after exiting MRI Protection Mode. (See Enable/Disable Beeper (EMBLEM S-ICD devices) on page 52.)*

Connecting and Disconnecting from the S-ICD Pulse Generator

This section provides the information necessary for selecting, connecting to, and disconnecting from the pulse generator.

WARNING: During a device software update, tachycardia therapy is suspended. Always monitor the patient and have external defibrillation equipment available during interrogation.

Caution: Use only the designated Boston Scientific S-ICD Programmer and appropriate software application to communicate with and program the S-ICD pulse generator.

Scanning for Pulse Generators

1. Select the Scan For Devices button on the Programmer start-up screen (Figure 2 on page 16). The Scan Progress Bar is displayed during the scanning process, at the conclusion of which the Device List screen appears. Select the Cancel button at any time to end the scanning process.

- When the scanning process is complete, a list of all pulse generators detected (up to 16) will be displayed on the Device List screen (Figure 7 on page 26). The devices that are in Shelf mode will be displayed as “Not Implanted.” Any devices that were previously taken out of Shelf mode are displayed either as “Implanted” or with the stored patient name.

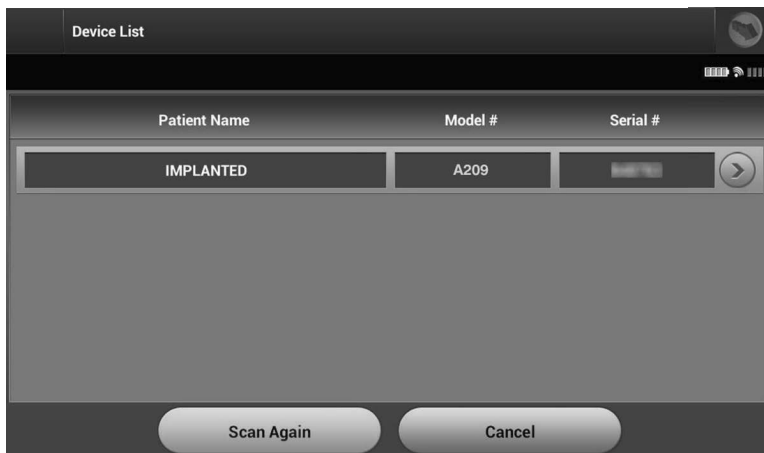


Figure 7: Device List screen (scrollable list)

- If the desired pulse generator is not listed, select the Scan Again button to re-initiate the scanning process. Select the Cancel button to return to the Programmer start-up screen.

Note: Refer to the *Inability to Communicate With the Pulse Generator* heading within the *Troubleshooting* section for further assistance.

Connecting to a Pulse Generator

Select the desired pulse generator from the Device List screen (Figure 7) to initiate the communication session.

Note: Regardless of how many pulse generators are located by a scan, the user must select a specific pulse generator from the list in order to begin active communication.

Connecting to a Pulse Generator in Shelf Mode

- The Programmer connects to the selected pulse generator after the selection is made. A window will appear indicating connection is in process.

2. The Device Identification screen appears once communication is established with the pulse generator.

Note: *The Device Identification screen is visible only while connecting to a pulse generator in Shelf mode.*

3. The device model and serial numbers are automatically acquired and displayed during the initial scanning process. Select Continue to remove the device from Shelf mode and prepare for implantation, or select Cancel to return to the Device List screen.

Connecting to an Implanted Pulse Generator

If an implanted pulse generator is chosen from the Device List screen, the following connection sequence occurs:

1. The Programmer connects to the selected pulse generator after the selection is made. A window will appear indicating connection is in process.

During the connection process informational messages and warning messages may display e.g. Smart Pass has been disabled.

2. The Device Status screen appears once communication is established with the pulse generator.

WARNING: During a device software update, tachycardia therapy is suspended. Always monitor the patient and have external defibrillation equipment available during interrogation.

Ending a Patient Session

To end an Online patient session and return the Programmer to its Offline operation mode:

1. Select the Main Menu icon on the Navigation Bar. The Main Menu screen appears.
2. Select the End Session button (Figure 8 on page 28).



Figure 8: Main Menu screen

3. Warning messages (Figure 9 on page 29) will be displayed to the user if any of the following conditions exist:
 - Therapy mode is programmed to Off
 - Reference S-ECG has not been acquired
 - Automatic or Manual Setup has not been completed
 - Optimization has not been completed. This message is displayed if Setup Optimization was not performed during the Automatic Setup process
 - SMART Pass is not enabled



Figure 9: Session incomplete message

4. Select the Continue button to end the patient session and return to the Programmer start-up screen, or select Cancel to remain Online and return to the Main Menu screen.

Note: *Once the Continue button is selected, the session is stored and communication is terminated.*

Note: *A telemetry session must be terminated using the End Session process as described in steps 1 through 4 above in order for data obtained during that session to be saved. If the Programmer is powered off during a session, either automatically or manually, session data will not be saved.*

Note: *In order to confirm that Therapy Mode is set to On upon disconnection, always use the End Session process and review all displayed warning messages.*

Programming the Pulse Generator at Implant

This section provides the information necessary for programming the pulse generator during an implant.

WARNING: During a device software update, tachycardia therapy is suspended. Always monitor the patient and have external defibrillation equipment available during interrogation.

Caution: Allow the pulse generator to reach an operating temperature range 25° C-45° C (77° F-113° F) before using telemetry communication

capabilities, programming, or implanting the pulse generator because temperature extremes may affect initial device function.

Caution: Use only the Model 3203 telemetry wand with the Programmer.

Caution: The wand is a non-sterile device. Do not sterilize the wand. The wand must be contained in a sterile barrier before use in the sterile field.

Caution: The Programmer is non-sterile and cannot be sterilized. It must remain outside the sterile field.

Caution: Confirm that the Programmer is in communication with the intended implanted S-ICD pulse generator.

Entering Electrode Information

The Programmer maintains information on the implanted electrode. To record this information for a patient's new or replacement electrode:

1. Select the Main Menu icon.
2. Select the Implant button.
3. Select the Automatic Setup icon in the Navigation Bar. The Automatic Setup screen appears (Figure 12 on page 33).
4. Select Set Electrode ID button.

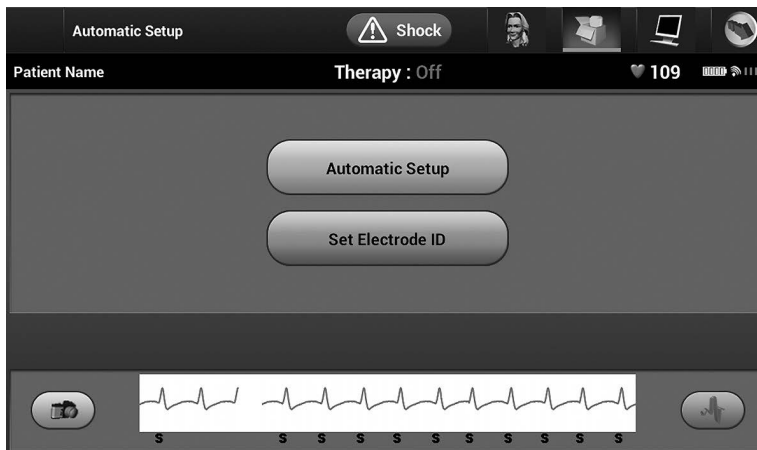


Figure 10: Select the Set Electrode ID button to enter electrode information

Note: ECG and heart rate information is not present on the Automatic Setup and Electrode ID setup screens until the electrode has been connected to the pulse generator.

5. Enter the electrode model and serial number.
6. Select the Program button to save the information. A confirmation screen will appear during communication with the device. Select Cancel to cancel information storage and return to the Automatic Setup screen.

Creating the Patient Chart

This chart contains reference information for the patient. To set up the patient chart:

1. Select the Main Menu icon on the Navigation Bar.
2. Select the Implant button.
3. Select the Patient View icon to access the Patient View screen (Figure 11).
4. The pulse generator model and serial numbers appear on the first line of the chart. The electrode model and serial numbers appear on the second line of the chart. The implant date appears on the third line of the chart. Using the on-screen keyboard, enter the following patient information:
 - Patient Name: up to 25 characters
 - Doctor Name: up to 25 characters

- Doctor Info: up to 25 characters
- Notes: up to 100 characters

The screenshot shows the 'Patient View' screen. At the top, there is a 'Patient View' header and a 'Shock' button with a warning icon. Below the header, there are icons for a patient, a printer, a laptop, and a globe. The main content area is divided into several sections:

- Patient Name:** A text input field.
- Therapy:** A status indicator showing 'On'.
- Device Model#:** A219
- Serial #:** [Redacted]
- Electrode Model#:** 1234
- Serial #:** [Redacted]
- Implant Date:** 07/18/2018
- Remaining Battery Life to ERI:** 100%
- Patient Name:** A text input field.
- Doctor Name:** A text input field.
- Doctor Info:** A text input field.
- Notes:** A large text input area.

At the bottom of the screen, there is a large 'Save' button.

Figure 11: Patient View screen

Note: The Notes field will automatically wrap the text with the presence of a space between any characters within the first line.

5. Select the Save button to update the pulse generator with the patient information.

Note: Failure to save the new patient information will result in loss of the entered data.

Automatic Setup

Before the S-ICD device can be activated, it must go through an initial Automatic Setup Process at the time of the implant.

The Automatic Setup Process is initiated as follows:

1. Select the Main Menu icon.
2. Select the Implant button.

3. Select the Automatic Setup icon on the navigation bar. The Automatic Setup screen appears. Select the Automatic Setup button on this screen to advance to the next screen.
4. Select Continue if the patient's heart rate is less than 130 bpm (Figure 12). For rates greater than 130 bpm, select the Cancel button and refer to Manual Setup on page 52.

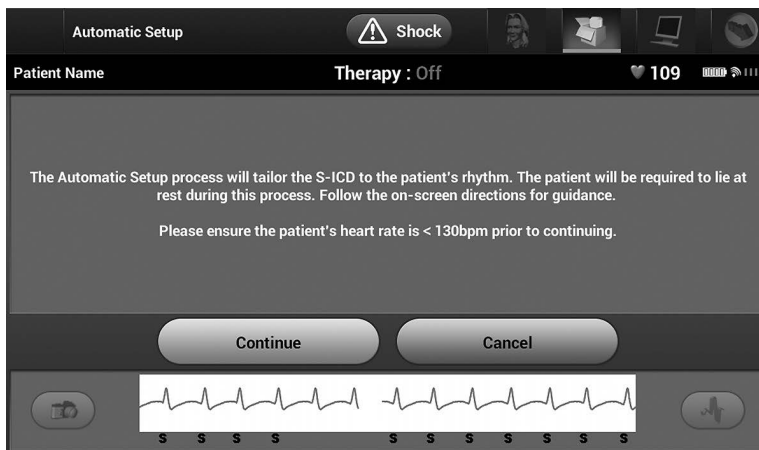


Figure 12: Automatic Setup screen

5. Once started the Automatic Setup will:
 - Perform the shock electrode integrity check to measure electrode impedance. Normal sub-threshold impedance range is < 400 Ohms.
 - Select the best sensing configuration. SMART Pass will be automatically configured based on the amplitude of the ECG signals in the selected vector. The sense electrode configuration appears on the printed report and can be viewed via the Manual Setup process. The status of SMART Pass (On/Off) is displayed on the SMART Settings Programmer screen and Summary Report. (For more information on SMART Charge and SMART Pass, see SMART Settings on page 54.)
 - Select the appropriate gain setting. The selected sense gain appears on the Printed Report and can be viewed via the Manual Setup process.

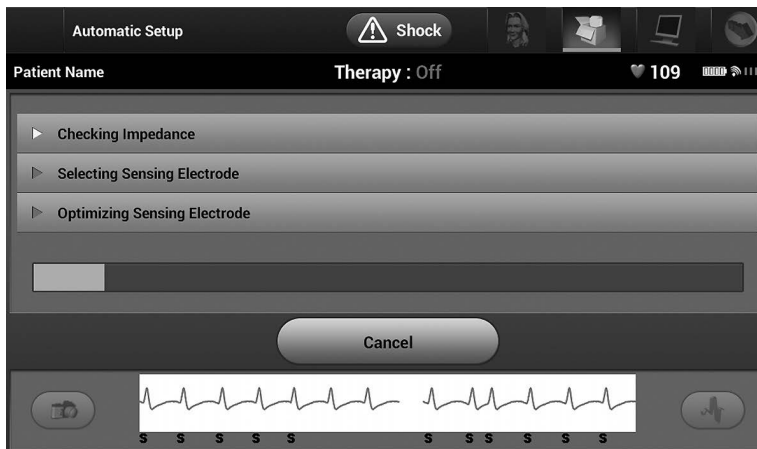


Figure 13: Measuring electrode impedance

The overall progress of the Automatic Setup process is shown in the status bar (Figure 13). When each function is complete, the arrow next to the function moves to a down position.

6. The Automatic Setup optimization process will be initiated. The Programmer will display a message requesting that the patient sit up. If Automatic Setup is being performed during implant, or if the patient is unable to sit up for some other reason, this step can be omitted by selecting the Skip button. If desired, Automatic Setup can be repeated during a follow-up session to include the optimization step.
7. Select the Continue button to finish the Automatic Setup process. A confirmation screen will appear when Automatic Setup is complete.
8. Following the optional optimization process, the Acquire Reference S-ECG screen is displayed. Select the Continue button to acquire a reference S-ECG.
9. Once the Reference S-ECG acquisition process begins, a status screen appears. The process may take up to one minute, during which the patient should remain still. During this process, a template of the patient's baseline QRS complex is stored in the pulse generator. Select Cancel at any time to end Reference S-ECG acquisition. When acquisition is complete, select the Continue button.

Programming Therapy Parameters

Once Automatic Setup has been completed, the pulse generator therapy parameters may be selected.

To set the therapy parameters:

1. Select the Main Menu icon on the Navigation Bar.
2. Select the Implant button.
3. Select the Device Settings icon on the Navigation Bar to display the Device Settings screen (Figure 14 on page 35).

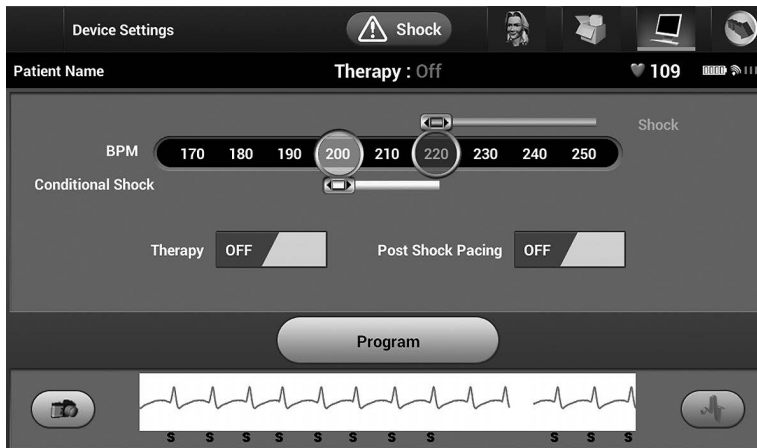


Figure 14: Device Settings screen

4. Select the desired therapy mode using the On/Off Therapy switch.
5. Select and drag the Conditional Shock Zone (yellow) and Shock Zone (red) slider bars to set the desired zone configuration.

Note: *The S-ICD System Clinical Investigation demonstrated a significant reduction in inappropriate therapy with the activation of the Conditional Shock Zone prior to hospital discharge (see the S-ICD pulse generator user's manual for a summary of the Clinical Investigation).*

- The Shock Zone is programmable between 170 and 250 bpm in steps of 10 bpm.
- The Conditional Shock Zone is programmable between 170 and 240 bpm in steps of 10 bpm. Enhanced detection criteria are automatically enabled when the Conditional Shock Zone is programmed.

- When programming both the Shock Zone and Conditional Shock Zone, maintain at least a 10 bpm difference between the two zones. If the Conditional Shock Zone slider (yellow) is dragged over the Shock Zone slider (red), the two sliders will merge to create a single Shock Zone.
6. If post-shock pacing is desired, set the Post Shock Pacing switch to the On position. (Post-shock bradycardia pacing occurs at a non-programmable rate of 50 bpm for up to 30 seconds. Pacing is inhibited if the intrinsic rate is greater than 50 bpm.)
 7. Select the Program button to apply the changes and program the pulse generator. A message confirming that the pulse generator settings were successfully programmed will appear. Select the Continue button to return to the Device Settings screen.
 8. If the pulse generator does not accept the programming, a message with instructions will appear on the Device Settings screen. Press the Continue button after following the instructions.

WARNING: The presence of other equipment operating in the same frequency bands used by the Programmer (402-405 MHz for the pulse generator and 2.4 GHz for the printer) may interfere with communication. Interference can occur even if the other equipment complies with the International Special Committee on Radio Interference (CISPR) emission requirements. This RF interference can be reduced by increasing the distance between the interfering device and the Programmer and pulse generator or printer. If communication problems persist, refer to the Troubleshooting section of this manual.

9. Once programming is confirmed, select the Continue button to proceed to the next operation.

Note: *The Pending Program Changes screen will appear if changes made to pulse generator settings on the Device Settings screen were not successfully applied to the pulse generator. Select Cancel to return to the Device Settings screen and save all settings changes, or Continue to abandon all pulse generator setting changes.*

Defibrillation Testing

Once the pulse generator is implanted and Therapy Mode is programmed On, defibrillation testing may be conducted. Prior to arrhythmia induction during implant procedure, the following recommendations for arm positioning are intended to reduce the potential for injury of the clavicle, arm and shoulder in the event of forceful muscle contraction:

- Avoid tight strapping of the arm to the arm board, and consider loosening arm restraints.
- Remove any wedge elevation below the torso, if used during the implant procedure, taking care to preserve the sterile field.

- Create a smaller angle of arm abduction from the torso by adducting the arm as close to the torso as feasible, taking care to preserve the sterile field. Temporarily place the hand in a neutral position while the arm is in a more adducted position, reverting to a supinated position if the arm needs to be abducted again.

WARNING: During arrhythmia induction, the induction current and subsequent shock may result in forceful contraction of the pectoralis major muscle which can exert significant acute forces on the glenohumeral joint as well as on the clavicle. This, in conjunction with a tightly restrained arm, may result in injury to the clavicle, shoulder, and arm, including dislocation and fracture.

WARNING: Always have external defibrillation equipment and medical personnel skilled in CPR available during implant and follow-up testing. If not terminated in a timely fashion, an induced ventricular tachyarrhythmia can result in the patient's death.

Caution: Successful VF or VT conversion during arrhythmia conversion testing is no assurance that conversion will occur post-operatively. Be aware that changes in the patient's condition, drug regimen, and other factors may change the DFT, which may result in nonconversion of the arrhythmia post-operatively. Verify with a conversion test that the patient's tachyarrhythmias can be detected and terminated by the pulse generator system if the patient's status has changed or parameters have been reprogrammed.

Note: *Defibrillation testing is recommended at implant, replacement, and concomitant device implants to confirm the ability of the S-ICD System to sense and convert VF.*

Note: *When the Hold to Induce button is pressed during defibrillation testing, the Programmer begins capturing the episode data generated during the test. This data is available for viewing and printing. (See Capturing and viewing S-ECG Strips on page 47 and Captured S-ECG Report on page 42.)*

To induce VF and test the S-ICD System:

1. Select the Main Menu icon on the Navigation Bar to access the Main Menu.
2. Select the Shock Test button to setup the induction test (Figure 15).
3. Select either standard (STD) or reverse (REV) polarity.
4. Select and drag the red marker to set the desired shock energy for the first delivered shock. The shock energy may be programmed from 10 to 80 J. A 15 J safety margin is recommended for defibrillation testing.

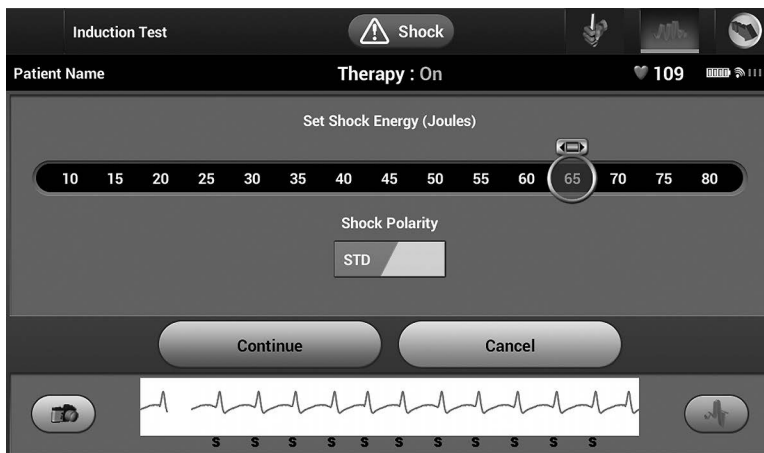


Figure 15: Setting the desired first shock energy for defibrillation testing

5. Select the Continue button to display the next Induction Test screen or select the Cancel button to return to the Main Menu screen.

Note: *Ensure that noise markers ("N") are not present on the S-ECG prior to induction. The presence of noise markers may delay detection and therapy delivery.*

6. On the Induction Test screen, select the check box to enable the Hold to Induce button.
7. Select and hold the Hold To Induce button for the desired duration.

The following functions occur during the test:

- The S-ICD System induces ventricular fibrillation using 200 mA alternating current (AC) at 50 Hz. Induction continues until the Hold To Induce button is released (up to a maximum of 10 seconds per attempt).

Note: *If necessary, the induction can be terminated by disconnecting the wand from the Programmer.*

- Arrhythmia detection and the Live S-ECG are suspended during induction. Once the Hold to Induce button is released, the Programmer displays the patient's rhythm.
- Upon detection and confirmation of an induced arrhythmia, the S-ICD System automatically delivers a shock at the programmed energy output and polarity.

Note: *When in active communication with a pulse generator, the Programmer emits an audible notification to indicate that the pulse generator is preparing to deliver a shock, whether that shock is commanded or is in response to a detected arrhythmia. The notification continues until the shock is either delivered or aborted.*

- If the shock fails to convert the arrhythmia, re-detection occurs and subsequent shocks are delivered at the pulse generator's maximum energy output (80 J).

Note: *Evaluate the sensing markers during the induced rhythm that follows the release of the Hold To Induce button. The S-ICD System uses a lengthened rhythm detection period. Consistent tachy "T" markers indicate that tachyarrhythmia detection is occurring, and that capacitor charging is imminent. If a high degree of amplitude variation is noted during the arrhythmia, a slight delay may be expected prior to capacitor charging or shock delivery.*

Note: *The pulse generator can deliver a maximum of five shocks per episode. An 80 J rescue shock can be delivered at any time prior to therapy delivery by pressing the Rescue Shock icon.*

- The Programmer will start capturing S-ECG data when the Hold to Induce button is pressed. The Captured S-ECG will cover the time period six seconds before the button is pressed and up to 102 seconds afterwards, for a maximum total of 108 seconds. The Induction S-ECGs will be viewable and printable from the Captured S-ECG screen, labeled as "Induction S-ECG".
8. At any time prior to therapy delivery, the programmed energy may be aborted by selecting the red Abort button.
 9. Select the Exit button to return to the Main Menu screen.

Performing a Follow Up

For a complete guide on clinical follow-up, see the section on post implant follow-up procedures in the EMBLEM™ S-ICD User's Manual.

Printing Reports from the Programmer

Printing Reports

Patient reports can be printed before or after a patient session is ended. It is recommended that a final report be printed immediately following the implant procedure. There are three patient reports:

- Summary Report
- Captured S-ECG Report
- Episode Reports

To print patient reports from either an Online or Offline session:

1. Select the Main Menu icon to display the Main Menu screen.
2. Select the Print Reports button to display the Print Reports screen (Figure 16 on page 40).

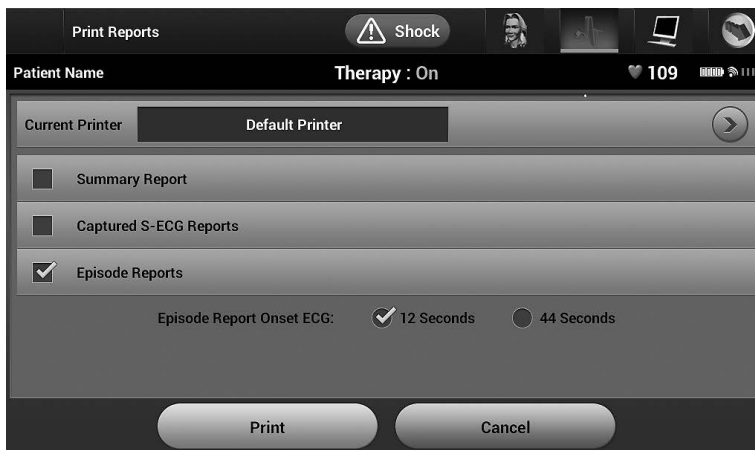


Figure 16: Print Reports screen

3. Select the desired report type. A check mark will appear next to the selected report. Report types are described below.
4. Select the Print button to print the selected report.
5. Select the Cancel button to return to the previously accessed screen.

Summary Report

To print a summary report, select the Summary Report option on the Print Reports screen and press the Print button. The report will print for either the current active session (if the Programmer is Online) or for the chosen stored session (if the Programmer is Offline.)

The Summary Report includes the following information:

- Printed Report Date
- Programmer Software Version
- Pulse generator Software Version
- Patient Name
- Date of Last Follow-Up
- Date of Current Follow-Up
- Implant Date
- Pulse generator Model/Serial Number
- Electrode Model/Serial Number
- Therapy Parameters
- SMART Charge Delay
- SMART Pass status (On/Off)
- Programmed Gain Settings and Sensing Configuration
- Pulse generator Integrity Check, if applicable
- Initial Shock Polarity Configuration
- Days with measured AF
- Estimate of measured AF
- Beeper Status (if disabled)
- MRI Information
- Episode Summary: Since Last Follow-Up and Since Initial Implant
- Battery Status
- Electrode Impedance Measurement

Captured S-ECG Report

To print a Captured S-ECG report:

1. Select the Captured S-ECG Reports option from the Print Reports screen.
2. A scrollable list containing both Captured S-ECG and Induction S-ECG strips is displayed (Figure 19). Select the desired S-ECG(s) to be printed by placing a check mark next to the selection(s).

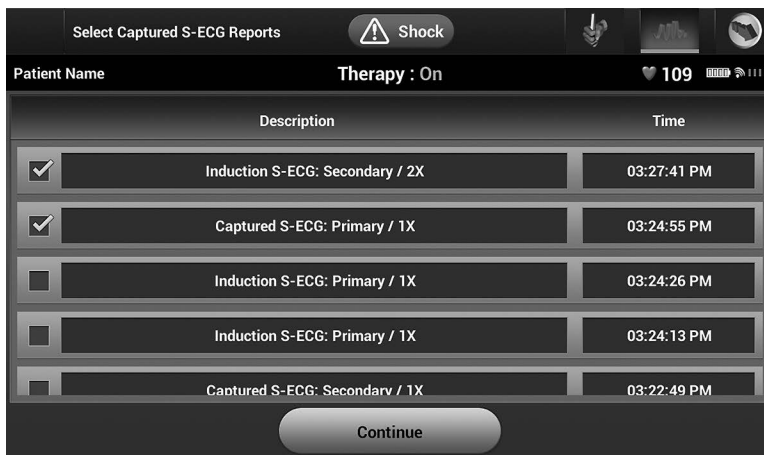


Figure 17: Captured S-ECG list (scrollable)

3. Select Continue to return to the Print Reports screen.
4. Select the Print button to print the selected report and return to the previously accessed screen.
5. Select the Cancel button to return to the previously accessed screen without printing the report.

Episode Reports

To print an Episode Report:

1. Select the Episode Reports option on the Print Reports screen.
2. The Select Episode Reports screen appears showing a list of the stored episodes (Figure 20). Select the episode(s) to be printed. A check mark appears next to the selected episode(s).

Note: In order to be available for printing, episodes must have been individually selected and viewed from the Episodes Screen (Figure 18 on page 43).



Figure 18: Select Episode Reports screen (scrollable)

3. Select Continue to return to the Print Reports screen. Either 12 seconds or 44 seconds of pre-episode S-ECG data may be selected using the radio buttons below the Episode Reports row. The default value for Episode Report Onset is 12 seconds.
4. Select the Print button to print the selected report and return to the previously accessed screen.
5. Select the Cancel button to return to the previously accessed screen without printing the report.

Export Patient Data

Patient data saved on the Programmer may be exported to a desktop or notebook computer using either of two means: wirelessly, over a pre-configured *Bluetooth*® pairing, or with a Model 3205 microSD™ log data card. (For information about *Bluetooth*® pairing between the Programmer and a desktop or laptop computer, see *Bluetooth*® Data Export on page 21.)

Export using *Bluetooth*® wireless technology

Ensure that the Programmer and the intended recipient computer are within 10 meters (33 feet) of each other before attempting a *Bluetooth*® wireless data transfer.

1. Select the Export Programmer Data button on the Programmer start-up screen. The Export Programmer Data Over Bluetooth® screen will appear.
2. Select one of the three export options (Export Today's Data, Export Last Seven Days, Export All). The "Select a receiving computer" pop-up window will appear.

Note: *The Export Today's Data and Export Last Seven Days options typically take less time to complete than the Export All option.*

3. The pop-up contains a scrollable list of all of the computers the Programmer has been paired with. Select the intended receiving computer from the list to begin the transfer.

Note: *Although every paired computer is listed in the scrollable box, only those within 10 meters (33 feet) of the Programmer can participate in a file transfer. Once one of the three export options has been selected, the Programmer will prepare the file transfer package and attempt the wireless transfer. An error message will appear if the transfer cannot be completed. Should this occur, move the Programmer to within 10 meters of the intended receiving computer or choose another computer within that distance. Restart the export process by selecting one of the three export options on the Export Programmer Data Over Bluetooth® screen.*

Export using a microSD™ card

Data may also be exported using a microSD™ card. For security reasons, the Programmer will only export data to Model 3205 microSD™ log data cards. Using any other microSD™ card will cause an error message (invalid card) to appear.

1. Navigate to the Programmer start-up screen.
2. Insert the card into the microSD™ slot according to the instructions presented in **Appendix A: Insertion and Removal of the microSD™ Card**. The instructions are also provided with Model 3205 microSD™ log data card. A Copy Data screen will appear when the microSD™ card has been properly inserted and recognized.

Note: *An invalid card error message will appear if any card other than a Model 3205 microSD™ log data card is inserted. The message may also appear if the Programmer does not recognize the Model 3205 card after insertion. Should this occur, remove the card and select the OK button on the error screen. Wait for the Programmer start-up screen to reappear and then reinsert the card.*

3. Select the Copy Data button on this screen and the next screen.
4. A confirmation screen is presented when the copy process is complete. Selecting the OK button will return the Programmer to its start-up screen.
5. Remove the microSD™ card according to the instruction sheet (Appendix A).

Note: *Navigating away from and returning to the start-up screen with the microSD™ card still inserted will cause the export process to begin again.*




S-ECG Features

The Programmer provides the capability to view, adjust and capture the streaming S-ECG from the pulse generator.

S-ECG Rhythm Strip Markers

The system provides annotations to identify specific events on the S-ECG. These markers are shown in Table 2: S-ECG Markers on Programmer Display Screens and Printed Reports on page 46.

Table 2: S-ECG Markers on Programmer Display Screens and Printed Reports

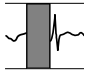Description	Marker
Charging ^a	C
Sensed Beat	S
Noisy Beat	N
Paced Beat	P
Tachy Detection	T
Discard Beat	•
Return to NSR ^a	
Shock	
Episode data compressed or not available	

^a Marker present on printed report but not on Programmer display screen.

S-ECG Scale Settings

To adjust the real-time S-ECG amplitude and display speed scale settings:

1. Select the S-ECG Display Settings icon located to the right of the Live S-ECG window. The S-ECG Settings screen is displayed.
2. Select and drag the Gain or Sweep Speed Scale bars as desired (Figure 21). The S-ECG scale will change according to the selected setting. The gain setting controls the visual gain. The Programmer defaults to Full Range for pulse generators with a gain setting of 1x and to 1/2 Range for pulse generators with a gain setting of 2x. The Sweep Speed slider controls the display speed of the scrolling Live S-ECG. The nominal sweep speed setting is 25 mm/sec.

Note: Amplitude settings and display speed adjustments on scrolling real-time S-ECG and Captured S-ECGs affect the display screen settings only and have no impact on the pulse generator settings for sensing.

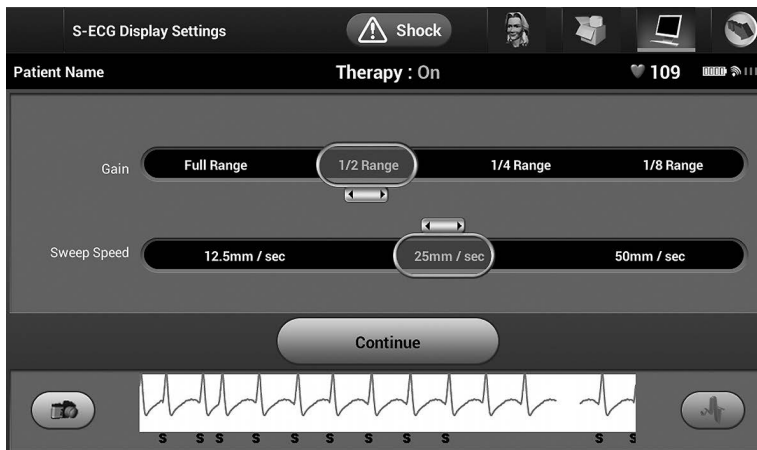


Figure 19: Adjusting Gain and Sweep Speed

Capturing and viewing S-ECG Strips

The Programmer can display and store real-time S-ECG rhythm strips. The Programmer saves a maximum of 50 recordings generated by:

1. Manually-captured twelve-second S-ECGs using the Capture S-ECG button which include:
 - 8.5 seconds before activation of the Capture S-ECG button
 - 3.5 seconds after activation of the Capture S-ECG button
2. S-ECGs captured automatically during induction testing which include:
 - 6 seconds before the induce button is pressed
 - Up to 102 seconds after the induce button is pressed

Note: The S-ICD suspends detection of sensed events for 1.6 seconds after a shock has been delivered. As a result, the S-ECG rhythm strip will not contain event markers during this 1.6 second post-shock interval.

If an additional recording is required, then the oldest previous recording is replaced with the new recording.

To manually capture a new S-ECG rhythm strip:

1. Select the Capture S-ECG button located to the left of the Live S-ECG window. The S-ECG will scroll across the display screen. Calipers appear below the Captured S-ECG rhythm strip. Each 12-second recording is date and time stamped according to the Programmer's date and time setting.

Note: *Induction S-ECGs are automatically generated during induction testing without additional user input.*

2. Select and move the calipers across the S-ECG strip to measure intervals as desired.
3. Select the Continue button to return to the previously accessed screen.

It is also possible to capture S-ECGs corresponding to all three sense vectors (Primary, Secondary, and Alternate) by using the Capture All Sense Vectors button on the Utilities screen (Figure 20 on page 49).

Viewing previously captured S-ECGs

When the Programmer is Online:

1. Select the Main Menu icon.
2. Select the Follow Up button.
3. Select the Captured and Stored Episode S-ECG icon. The Captured S-ECG screen appears.
4. Select one Captured S-ECG or Induction S-ECG from the list. The S-ECG details screen appears.
5. Select and drag the calipers to view details.
6. Select the Continue button to return to the Captured S-ECG list screen.

When the Programmer is Offline:

1. Select the Stored Patient Sessions button from either the Programmer start-up screen or the Main Menu.
2. Select the desired stored patient session.
3. Select one Captured S-ECG from the list. The Captured S-ECG Details screen appears.

Note: *Not all stored patient sessions contain captured S-ECGs. A message to that effect is presented when such patient sessions are opened. In this event select the Main Menu icon, then select the End Session button. This action returns you to the Programmer start-up screen.*

4. Select and drag the calipers to view details.
5. Select the Continue button to return to the Captured S-ECG list screen.

Utilities Screen

The Programmer Utilities screen provides access to additional device features. These may include Acquire Reference S-ECG, Capture All Sense Vectors, Beeper Control, Manual Setup, SMART Settings, and AF Monitor.

To access the Utilities screen during an Online session:

1. Select the Main Menu icon to display the Main Menu screen.
2. Select the Utilities button. The Utilities screen appears (Figure 20 on page 49).

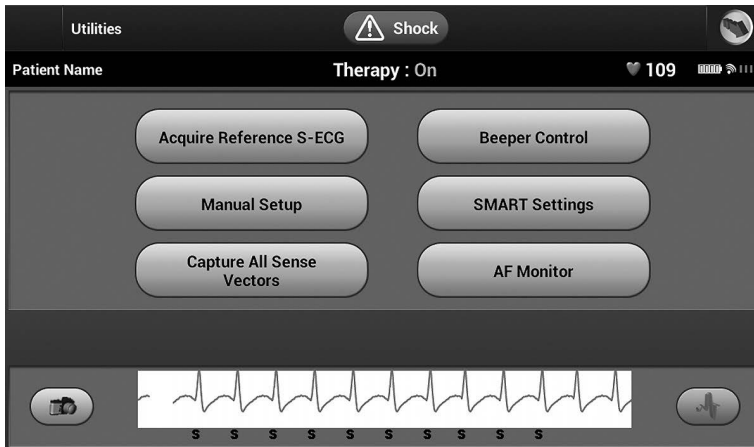


Figure 20: Utilities screen

Acquire Reference S-ECG

To acquire a manual Reference S-ECG:

1. From the Utilities screen (accessible from the Main Menu screen), select the Acquire Reference S-ECG button to access the Acquire Reference S-ECG screen.

2. Select Continue to acquire a Reference S-ECG. The Programmer will begin acquiring the Reference S-ECG. A message will appear requesting that the patient remain still. The reference S-ECG QRS template is recorded and stored in the pulse generator.
3. Select the Continue button to complete the process and return to the Utilities screen. The Cancel button can be used at any time to end S-ECG acquisition and return to the Utilities screen.

Capture All Sense Vectors

The Capture All Sense Vectors button on the Utilities screen configures temporary Programmer settings that allow you to capture S-ECGs generated from each of the three sense vectors (Primary, Secondary, and Alternate). This process takes approximately one minute. The Programmer returns to its original settings configuration after all S-ECGs have been captured.

To capture the three sense vectors:

1. From the Utilities screen (accessible from the Main Menu screen), select the Capture All Sense Vectors button.
2. The Capturing 12 Second S-ECG screen will appear and display the status of the sense vector capture process.

Once captured, the three S-ECGs can be viewed by following the steps outlined in **Viewing previously captured S-ECGs** on page 48.

Beeper Control

The pulse generator has an internal warning system (beeper) that may emit an audible tone to alert the patient to certain device conditions that require prompt consultation with the physician. These conditions may include:

- Electrode impedance out of range
- Prolonged charge times
- Failed Device Integrity Check
- Irregular battery depletion

This internal warning system is automatically activated at time of implant. Once triggered, if the beeper is enabled, tones beep for 16 seconds every nine hours until the trigger condition has been resolved. If the triggering condition reoccurs, then the tones will once again alert the patient to consult the physician.

Caution: Patients should be advised to contact their physician immediately if they hear tones coming from their device.

Note: *Access to the Reset Beeper display screen is enabled only when an alert condition occurs. If an alert condition is activated, a notification screen will appear upon connection.*

WARNING: The beeper may no longer be usable following an MRI scan. Coming in contact with the strong magnetic field of an MRI scanner may cause a permanent loss of the beeper volume. This cannot be recovered, even after leaving the MR scan environment and exiting MRI Protection Mode. Before an MRI procedure is performed, a physician and patient should weigh the benefit of the MR procedure against the risk of losing the beeper. It is strongly recommended that patients are followed on LATITUDE NXT after an MRI scan if they are not already. Otherwise, an in-clinic follow-up schedule of every three months is strongly recommended to monitor device performance.

Reset Beeper

To reset the beeper, select the Beeper Control button from the Utilities screen (accessible from the Main Menu) to open the Set Beeper Function screen.

Select the Reset Beeper button to suspend audible beeping tones triggered by an alert condition. If the alert condition is not corrected, the audible beeping tones will be reactivated during the next automatic S-ICD System self-check.

Disable Beeper (SQ-Rx devices)

In SQ-Rx devices, Beeper Control allows for the deactivation of beeping on alert conditions (Disable Beeper). Perform the following steps to disable the beeper:

Note: *The Disable Beeper function is only available once device ERI or EOL is reached.*

1. From the Utilities screen, select Beeper Control to open the Set Beeper Function screen.
2. Select Disable Beeper to disable the beeper for the device.

Note: *This will permanently disable all beeping on alert conditions for the SQ-Rx device. However, this will not affect beeper functionality for when a magnet is placed on the device or when a Programmer connects to the device.*

Enable/Disable Beeper (EMBLEM S-ICD devices)

In EMBLEM S-ICD devices, the beeper must be tested before being enabled or disabled. Perform the following steps to test the beeper:

Note: For EMBLEM S-ICD devices, the Test Beeper function is only available when beeping for an alert condition is not active.

1. From the Utilities screen, select Beeper Control.
2. Select the Test Beeper button from the Set Beeper Function screen.
3. Evaluate if the beeper is audible using a stethoscope.
4. If the beeper is audible, select the Yes, Enable Beeper button. If the beeper is not audible or you wish to permanently disable beeping functionality, select the No, Disable Beeper button.

CAUTION: In the Programmer steps to program and test the Beeper, selecting “No, Disable Beeper” will disable beeping functionality for alert conditions, when a magnet is placed over the device, and when the S-ICD application connects to the device.

If the beeper is not audible to the patient, it is strongly recommended that the patient has a follow-up schedule of every three months either on LATITUDE NXT or in-clinic to monitor device performance.

For additional information regarding the beeper, refer to the MRI Technical Guide or contact Boston Scientific using the information on the back cover.

Manual Setup

Manual Setup enables the user to perform the electrode integrity test and select the electrode sensing configuration and gain setting in the pulse generator. During Manual Setup, the system will also automatically enable SMART Pass if appropriate.

1. From the Utilities screen (accessible from the Main Menu screen), select the Manual Setup button. The Measure Impedance screen appears.
2. Select the Test button to perform the electrode integrity test.
3. Select the Continue button.
4. There are three available sensing vectors that can be manually selected from the Manual Setup screen (Figure 21 on page 53):
 - Primary: Sensing from the proximal electrode ring on the subcutaneous electrode to the surface of the active pulse generator

- **Secondary:** Sensing from the distal sensing electrode ring on the subcutaneous electrode to the surface of the active pulse generator
- **Alternate:** Sensing from the distal sensing electrode ring to the proximal sensing electrode ring on the subcutaneous electrode

The gain setting adjusts the sensed S-ECG signal sensitivity. It may be manually selected with the Select Gain switch on the Manual Setup screen.

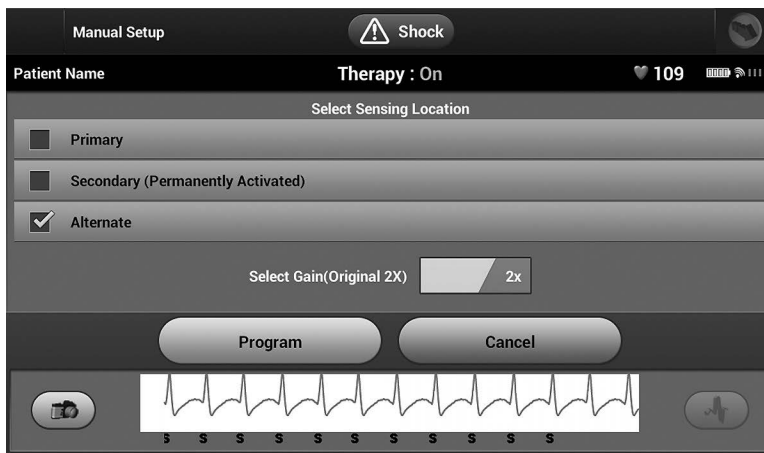


Figure 21: Manual Setup screen with sensing vector and gain options

- 1x Gain (± 4 mV): 1x Gain should be selected when the signal amplitude is large enough to cause clipping when the 2x gain is selected.
- 2x Gain (± 2 mV): 2x Gain should be selected when the signal amplitude is small enough to allow use of a more sensitive setting without causing clipping of the captured signal. The 2x gain selection amplifies the signal twice as much as the 1x gain selection.

Note: *The gain setting may affect certification phase performance. Specifically, using 2x gain may improve identification of noise.*

To program the manually selected sense configuration:

1. Select the Program button to save the sense vector and gain settings.

2. Select the Continue button. When the continue button is selected, the device will automatically evaluate if SMART Pass should be enabled. Refer to the S-ICD User's Manual for additional information about SMART Pass. For assistance, contact Boston Scientific using the information on the back cover.
3. The Acquire Reference S-ECG process is automatically enabled during the Manual Setup Process. Select the Continue button to acquire a reference S-ECG. A confirmation screen will appear when the captured reference S-ECG is acquired.

SMART Settings

The SMART Settings screen allows the user to access information and functions for the SMART Charge and SMART Pass features.

SMART Charge

Through the SMART Charge feature, the pulse generator charge initiation sequence adapts to the occurrence of non-sustained ventricular arrhythmia episodes by delaying capacitor charging. This conserves battery life and may prevent unnecessary shocks for non-sustained arrhythmias. Refer to the pulse generator manual for further information about the SMART Charge feature.

SMART Charge is enabled automatically when an untreated ventricular arrhythmia episode is recorded. Resetting returns the SMART Charge value to zero. To reset the SMART Charge feature:

1. From the Utilities screen (accessible from the Main Menu screen), select the SMART Settings button. The SMART Settings screen appears (Figure 22).

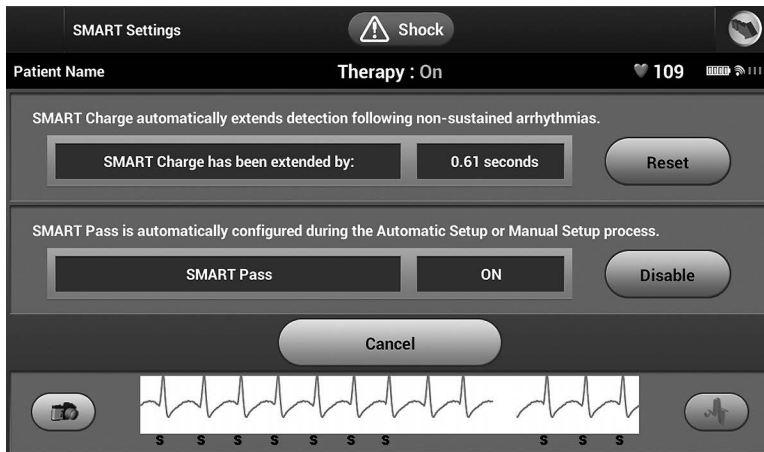


Figure 22: SMART Settings screen

2. Select the Reset button to reset the SMART Charge to zero or press Cancel to return to the Utilities screen without resetting the SMART Charge.
3. A confirmation window will appear with the message: "SMART Charge successfully reset."
4. Press the Continue button to return to the Utilities screen.

Disabling SMART Pass

The SMART Pass feature is designed to reduce oversensing while still maintaining an appropriate sensing margin. The device continuously monitors the ECG signal amplitude and automatically disables SMART Pass if under-sensing is suspected.

SMART Pass can be manually disabled if under-sensing is suspected by selecting the Disable button on the SMART Settings screen.

Note: *If SMART Pass is disabled, another automatic or manual setup must be performed to re-enable the feature.*

AF Monitor

The AF Monitor feature is designed to assist in the diagnosis of atrial fibrillation.

The AF Monitor feature can be enabled/disabled using the On/Off switch accessed through the AF Monitor button on the Utilities screen.

Note: *Print desired reports and/or save session data (via End Session) prior to programming AF Monitor Off. When AF Monitor is programmed Off, currently stored AF Monitor statistics are cleared and can no longer be printed or saved.*

Select the Program button to apply the changes and program the pulse generator.

The following statistics are available on the Programmer screen by selecting the AF Monitor button:

- **Days with measured AF:** Provides the number of days within the last 90 where AF was detected
- **Estimate of measured AF:** Provides the total percent of detected AF within the last 90 days

Refer to the S-ICD User's Manual for further information about AF Monitor.

Additional Programmer Functions

Rescue Shock

The Rescue Shock icon is available in the navigation bar on the Programmer display when the Setup Process is complete and a pulse generator is actively communicating with the Programmer. During active communication, a maximum (80 J) rescue shock can be delivered upon Programmer command.

To deliver a rescue shock:

1. Select the red Rescue Shock Icon at the top of the Programmer screen. The Rescue Shock screen appears (Figure 23).

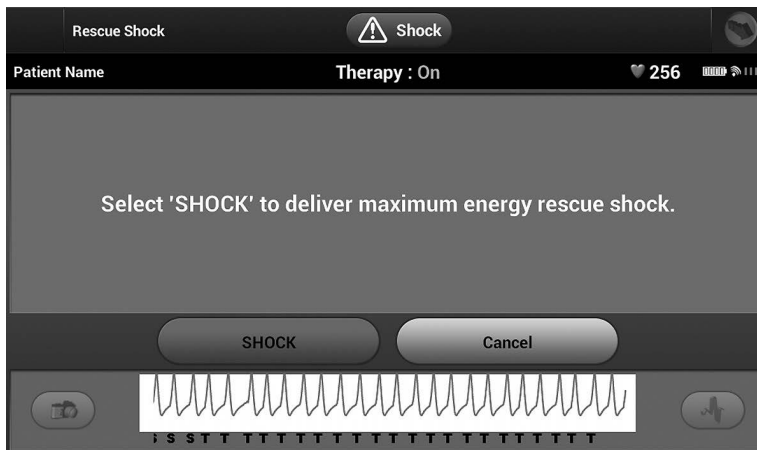


Figure 23: Rescue Shock screen

2. Select the Shock button to begin charging the pulse generator for a Rescue shock. A red background screen with the word “Charging” will appear. Selecting the Abort button will prevent delivery of a rescue shock and will return to the Device Settings screen.
3. A confirmation screen will appear with notification that the shock was delivered successfully along with the corresponding shock impedance.

WARNING: High shocking electrode impedance may reduce VT/VF conversion success.

Caution: A reported shock impedance value of greater than 110 ohms from a delivered shock could indicate sub-optimal system placement. Care should be taken to place both the pulse generator and electrode directly on the fascia without underlying adipose tissue. Adipose tissue can add significant impedance to the high voltage shock current pathway.

Caution: A reported shock impedance value of less than 25 ohms from a delivered shock could indicate a problem with the pulse generator. The delivered shock may have been compromised, and/or any future therapy from the pulse generator may be compromised. If a reported impedance value of less than 25 ohms is observed, correct functioning of the pulse generator should be verified.

If for any reason the shock could not be delivered, a red background screen will appear with a message stating “The shock could not be delivered.”

Note: *In the event telemetry is lost, pulse generator commands— including Rescue Shocks— will not be available until telemetry is reestablished.*

Manual Shock

Manual Shock allows the user to deliver a synchronized shock during a sinus rhythm, an atrial rhythm or a ventricular rhythm. The shock energy level is user-configured in the 10 to 80 joule range and the polarity is also user-configured (Figure 24 on page 58). Manual shock may also be utilized at a low energy to assess system impedance/integrity either at implant or as warranted by patient condition. A manual shock may be administered with the Therapy Mode set to On or Off.

To access Manual Shock, select the Shock Test button on the main menu. The Induction Test screen will appear. Select the Manual Shock icon in the navigation bar at the top of the screen to view the Manual Shock Test screen.

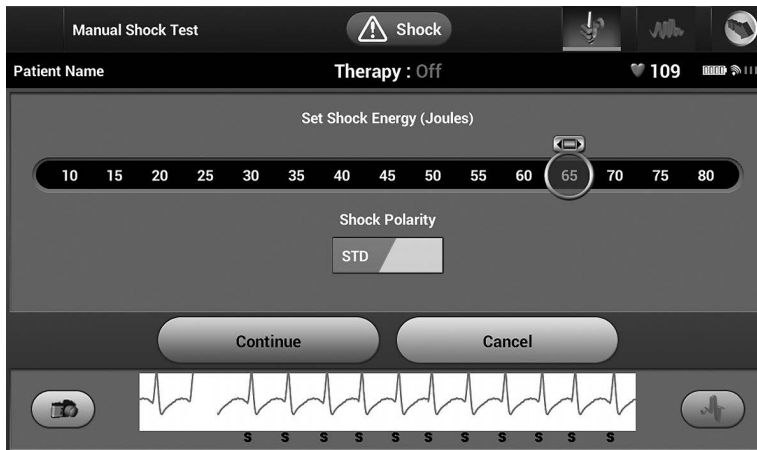


Figure 24: Manual shock

S-ICD System Magnet Use

A Boston Scientific magnet or Cameron Health magnet greater than 60 gauss is a non-sterile accessory that may be used to temporarily inhibit the delivery of therapy from the pulse generator.

For detailed information about using the magnet, refer to the appropriate S-ICD User's Manual.

Behaviors of magnet application include:

- Suspend arrhythmia detection and inhibit shock therapy delivery
- Terminate post-shock pacing therapy
- Prohibit arrhythmia induction testing
- Activate the pulse generator's beeper with each detected QRS complex for 60 seconds if beeper is turned on and is audible

WARNING: Use caution when placing a magnet over the S-ICD pulse generator because it suspends arrhythmia detection and therapy response. Removing the magnet resumes arrhythmia detection and therapy response.

WARNING: In patients with a deep implant placement (greater distance between the magnet and the pulse generator) magnet application may fail to elicit the magnet response. In this case the magnet cannot be used to inhibit therapy.

Caution: Do not place a magnet on the Programmer.

Note: *A Programmer commanded Rescue Shock can override the use of the magnet as long as the magnet was in place prior to the initiation of the programming command. If the magnet is applied after the initial command, the Rescue Shock will be terminated.*

Note: *Magnet application does not affect wireless communication between the pulse generator and the Programmer.*

MAINTENANCE

Charging the Programmer

When not in use, it is recommended that the Programmer remain connected to the external power supply, which in turn is connected to the AC mains. This will ensure that the internal battery is adequately charged.

Cleaning the Programmer and Wand

Keep the Programmer away from dust and dirt. Do not use harsh chemicals, cleaning solvents, or strong detergents to clean the Programmer and wand.

To clean the Programmer and wand as needed:

1. Turn the Programmer off.
2. Gently wipe the Programmer screen with a soft, clean, dry cloth.
3. Clean the Programmer plastic case and the wand by wiping them with an isopropyl alcohol-moistened cloth.
4. Dry the Programmer immediately to remove residue.

Service

There are no user-accessible or user-serviceable parts or components in the Programmer. If any service, repair, or replacement of internal components is needed, the Programmer must be returned to Boston Scientific. For instructions and return packaging, contact Boston Scientific using the information on the back cover of this manual.

When requesting service, please provide information concerning the nature of the failure and the manner in which the equipment was used when the failure occurred. The model number and serial number should also be provided.

Maintenance Check

Prior to each use, you should perform a visual inspection and verify the following:

- Mechanical and functional integrity of the Programmer, cables, and accessories.
- Legibility and adherence of the Programmer labels.
- That the Programmer start-up screen appears a few seconds after you turn on the Programmer. (The normal power-up process verifies that the Programmer has passed its internal checks and is ready for use.)

Safety Measurements

National regulations may require that the user, manufacturer, or manufacturer representative periodically perform and document safety tests of the Programmer. If such testing is required in your country, follow the testing interval and extent of testing as regulated in your country. If you do not know the national regulations in your country, please contact Boston Scientific using the information on the back cover of this manual. If IEC/EN 62353 is a required standard in your country, but no specific testing or interval is specified, it is recommended that you perform these safety tests using the direct method as specified in IEC/EN 62353 at an interval of every 24 months. Test values are shown in the Nominal Specifications table (Table 11).

Disposal of Programmer and Accessories

The Programmer and accessories are designed to provide years of service under typical use. When no longer needed:

- Return or exchange a Programmer by contacting Boston Scientific using the information on the back cover of this manual. **Do not dispose of the Programmer in the trash or at electronics recycling facilities as it may contain sensitive personal information.**
- Dispose of accessories (wand and power cord) in a separate collection for electrical and electronic equipment. Do not place accessories in the trash.
- Dispose of digital data storage media, such as microSD™ memory cards, in accordance with applicable privacy and security policies and regulations.

TROUBLESHOOTING

This section presents potential Programmer issues and the possible solutions. Of note, restarting the Programmer can often resolve many of the issues listed below. The Programmer can be restarted by pressing and holding the power button until the system shutdown menu appears and then choosing the “Restart” option.

Contact Boston Scientific using the information on the back cover of this manual for additional assistance.

Inability to Print

If unable to print, follow the steps below:

1. Ensure that the printer is turned On and that it contains paper and a sufficient ink supply.
2. Check printer feed for paper jam.
3. Ensure, as applicable, that the wireless function is enabled on the printer or that the *Bluetooth*® wireless adapter is fully inserted into the USB slot on the printer.

No Printer Available

The No Printer Available screen will appear if a printer was not set up. Select the Try Again button or refer to the Printer Selection section for instructions.

Touch Screen Inactive while Connected to AC Power

If the touch screen does not function while the Programmer is connected AC power via the external power supply, disconnect and reconnect the external power supply and restart the Programmer.

Loss of Communication with Printer

When communication between the Programmer and the printer fails, a Printing Error screen will appear with a message stating "Error while printing reports. Press 'Continue' to try printing any remaining reports, or 'Cancel' to cancel the current print job."

If this occurs:

1. Select the Try Again button to reconnect to the printer.
2. Ensure, as applicable, that the wireless function is enabled on the printer or that the *Bluetooth*® wireless adapter is fully inserted into the USB slot on the printer.
3. Move the Programmer closer to the printer.
4. Move any devices and the associated cables that may be interfering with the RF communication.

Inability to Communicate With the Pulse Generator

If the Programmer is unable to communicate with the pulse generator, follow the steps below:

1. Attempt to reposition the wand.
2. Select Scan For Devices from the Programmer start-up screen or select Scan Again from the Device List screen to locate the desired device.
3. Move any equipment and associated cables that may be interfering with RF communication.
4. If available, attempt to communicate using a different S-ICD System Programmer and/or wand.
5. Apply a pulse generator magnet to the pulse generator to elicit beeper tones. Remove the magnet and re-attempt communication.

COMPLIANCE STATEMENTS

EMI/RFI

This equipment has been tested and found to comply with the applicable limits for medical devices, IEC 60601-1-2:2014 or Active Implantable Medical Device Directive 90/385/EEC.

Although this testing shows the device to provide reasonable protection against harmful interference in a typical medical installation, there is no guarantee that interference will not occur in a particular installation. If the device does cause harmful interference the user is encouraged to try and correct the interference by the following measures:

- Reorient or relocate the device
- Increase the separation between the devices
- Connect the equipment to an outlet on a different circuit
- Contact Boston Scientific using the information on the back cover of this manual.

Essential Performance

In order for the Model 3200 Programmer to meet its intended use, it must interrogate and maintain a communications link with an S-ICD pulse generator as well as being able to appropriately detect touch screen button presses. Therefore those functions that pertain to communications with the implanted cardioverter defibrillator and detection of touch screen presses are considered essential performance.

Federal Communications Commission (FCC) Compliance

FCC ID: ESCCRM320014

Contains: FCC ID ESCCRM70514

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
2. This device must accept any interference received, including interference that may cause undesired operation

This transmitter is authorized by rule under the Medical Device Radiocommunication Service (in part 95 of the FCC Rules) and must not cause harmful interference to stations operating in the 400.150–406.000 MHz band in the Meteorological Aids (i.e. transmitters and receivers used to communicate weather data), the Meteorological Satellite, or the Earth Exploration Satellite Services and must accept interference that may be caused by such stations, including interference that may cause undesired operation. This transmitter shall be used only in accordance with the FCC Rules governing the Medical Device Radiocommunication Service. Analog and digital voice communications are prohibited.

Although this transmitter has been approved by the Federal Communications Commission, there is no guarantee that it will not receive interference or that any particular transmission from this transmitter will be free from interference.

Caution: Changes or modifications not expressly approved by Boston Scientific could void the user's authority to operate the equipment.

Industry Canada (IC) Compliance

IC 4794A-CRM32004

Contains: IC 4794A-CRM7054

This device complies with Industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and
2. This device must accept any interference received, including interference that may cause undesired operation.

This device may not interfere with stations operating in the 400.150-406.000 MHz band in the meteorological aids, meteorological-satellite, and earth exploration-satellite services, and must accept any interference received, including interference that may cause undesired operation.

The electromagnetic emissions and immunity information is provided in Table 3: Guidance and Manufacturer's Declaration—Electromagnetic Compatibility on page 65.

DECLARATIONS TABLES

The EMBLEM S-ICD Programmer, Model 3200, is suitable for use in professional health care facility environments. The customer or the user of this system should assure that it is used in such an environment.

The emissions characteristics of this equipment make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required) this equipment might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the equipment.

Table 3: Guidance and Manufacturer’s Declaration—Electromagnetic Compatibility

Test	Compliance	Electromagnetic environment—guidance
Proximity fields from RF wireless communications equipment	CISPR 11 Group 1 Class A	The EMBLEM S-ICD Programmer, Model 3200, uses RF energy only for its intended uses in communication with the implanted device or connectivity functions. Its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
Protection of the public mains network	CISPR 11 Class A IEC 61000-3-2 IEC 61000-3-3	The EMBLEM S-ICD Programmer, Model 3200, is suitable for use in professional health care facility environments.
Electrostatic Discharge	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, and ± 15 kV air	
Radiated RF EM field	3 V/m from 80 MHz to 2.7 GHz	
Proximity fields from RF wireless communications equipment	380 - 390 MHz: 27 V/m 430 - 470 MHz: 28 V/m 704 - 787 MHz: 9 V/m 800 - 960 MHz: 28 V/m 1700 - 1900 MHz: 28 V/m 2400 - 2570 MHz: 28 V/m 5100 - 5800 MHz: 9 V/m	
Rated power frequency magnetic field	30 A/m	
Electrical fast transients/ bursts	± 2 kV input AC power ± 1 kV SIP/SOP	
Surges line-to-line	± 0.5 kV, ± 1 kV input AC power	
Surges line-to-ground	± 0.5 kV, ± 1 kV, ± 2 kV input AC power	

Table 3: Guidance and Manufacturer’s Declaration—Electromagnetic Compatibility

Test	Compliance	Electromagnetic environment—guidance
Conducted disturbances induced by RF fields	3 V/m from 0.15 MHz to 80 MHz 6 V/m in ISM bands from 0.15 MHz to 80 MHz	The ISM bands between 0.15 MHz and 80 MHz are 6.765 MHz to 6.795 MHz 13.553 MHz to 13.567 MHz 26.957 MHz to 27.283 MHz 40.66 MHz to 40.70 MHz. The amateur radio bands between 0.15 MHz and 80 MHz are: 1.8 MHz to 2.0 MHz 3.5 MHz to 4.0 MHz 5.3 MHz to 5.4 MHz 7.0 MHz to 7.3 MHz 10.1 MHz to 10.15 MHz 14.0 MHz to 14.2 MHz 18.07 MHz to 18.17 MHz 21.0 MHz to 21.4 MHz 24.89 MHz to 24.99 MHz 28.0 MHz to 29.7 MHz 50.0 MHz to 54.0 MHz.
Voltage dips ^a	0% U_t for 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0% U_t for 1 cycle and 70% U_t for 25/30 cycles at 0°	
Voltage interruptions ^a	0% U_t for 250/300 cycles	

a. Voltage dips and interruptions: U_t is the AC mains voltage prior to application of the test level

Table 4: EMI/RFI Information: Programmer-to-Pulse Generator Communication

Specification	Medical Implant Communications Service (MICS)
Frequency band	402-405 MHz
Modulation type	FSK
Radiated Power	<25 μ W
Bandwidth	< 300 KHz

Table 5: EMI/RFI Information: *Bluetooth*® Wireless Printing and Data Transfer

Specification	<i>Bluetooth</i> ® wireless technology
Frequency band	2.402-2.480 GHz
Modulation type	GFSK, $\pi/4$ -DQPSK, 8DPSK
Radiated power	<10 mW
Bandwidth	< 1.5 MHz

Table 6: Product Guidelines

Component	Requirement	
DC Power		
Battery pack type	4000 mAh 3.7 volt lithium-ion battery pack	
Charge time	Approximately 5 hours	
Power Supply		
Input	100 – 240 VAC, 50 – 60 Hz, 0.5 A	
Output	5.5 VDC, 3.64 A Power: 20 W	
Manufacturer/Model	Elpac Power Systems MWA020005A	
Environment	Operating	Storage and Transport
Temperature	15° C to +38° C (+59° F to +100° F)	-10° C to +55° C (+14° F to +131° F)
Relative humidity	5% to 93% maximum at 40° C, non-condensing	5% to 93% maximum at 40° C, non-condensing
Atmospheric pressure	50 kPa to 106 kPa (7.252 psi to 15.374 psi)	50 kPa to 106 kPa (7.252 psi to 15.374 psi)

SPECIFICATIONS

Table 7: Specifications

Parameter	Specification
Safety classification	Class I
Ingress protection rating	IPX0

Table 7: Specifications

Parameter	Specification
Dimensions Width x Depth x Height	24.0 cm x 12.7 cm x 2.6 cm 9.4 in x 5.0 in x 1.0 in
Weight	0.6 kg, 1.3 lbs
Standard Screen Display	WVGA, 1024 x 600 pixels, 16M TFT

Table 8: Nominal Specifications (with device connected to external power supply)

Characteristic	Nominal
Electrical Safety Testing – IEC 60601-1:2012	
Earth resistance	Not accessible
Earth leakage current	5 mA Normal Condition (NC)
	10 mA Single Fault Condition (SFC)
Patient leakage current	100 µA Normal Condition (NC)
	500 µA Single Fault Condition (SFC) (mains on applied parts)
Electrical Safety Testing – IEC 62353:2008 allowed values	
Protective Earth Resistance	Not accessible
Equipment leakage – direct method	500 µA
Patient leakage current – direct method (Wand, BF)	<= 5000 µA
Insulation resistance	Not accessible
Safety Features	
Defibrillator protection	To 5000 V, 400 J

DEFINITION OF PACKAGING LABEL AND DEVICE SYMBOLS

Table 9: Packaging and Device Symbols: Model 3200 Programmer

The following symbols may be used on the Model 3200 Programmer, its accessories, and their packaging.































Symbol	Specification	Symbol	Specification
	Follow instructions for use on this website: www.bostonscientific-elabeling.com		Type BF applied part
	Electrostatic sensitive device		Non-ionizing electromagnetic radiation
	Temperature limitation		Humidity limitation
	Atmospheric pressure limitation		Manufacturer
	Serial Number		Date of manufacture
	Reference number		Authorized representative in the European community
	Lot number		Non-sterile
	ACMA compliance mark		Australian sponsor address
	Power plug storage		Door, open
	Consult instructions for use on this website: www.bostonscientific-elabeling.com		Wand

Table 9: Packaging and Device Symbols: Model 3200 Programmer

The following symbols may be used on the Model 3200 Programmer, its accessories, and their packaging.



Symbol	Specification	Symbol	Specification
	Separate collection		Proper insertion of microSD™
R-NZ	New Zealand R-NZ RF compliance mark		External power supply port
MD	Medical device under EU legislation		MR unsafe
PN	Part Number		Programmer
	Contents		Power Cord
	Power supply		Literature enclosed
	Mark for nationally recognized testing for safety standards		

WARRANTY

Limited Warranty

A limited warranty may apply to this Programmer. For warranty eligibility and to obtain a copy of the limited warranty, contact Boston Scientific using the information on the back cover.


Model 3200 S-ICD Programmer

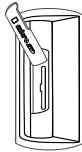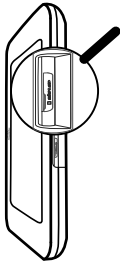


Lift the microSD™ slot cover and swing it away. Confirm that there is no card in the slot.



Place the microSD™ card into the empty slot with the lettering facing the same direction as the Programmer screen.

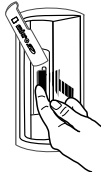

Gently press the card into the slot until it latches into place. There will be a slight resistance and a click, followed by a soft audible click which indicates that the latch is activated.

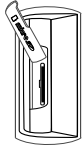


The properly mounted card will be flush with the slot opening surface.




Close the microSD™ slot cover.


Insert microSD™

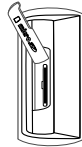


Lift the microSD™ slot cover and swing it away. Verify that a card is in the slot.

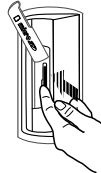


Press the edge of the card with your fingernail until the latch releases and you hear a click. Let the card gradually come out until you hear a second click, then remove your finger.

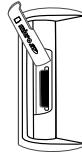

At this point it is safe to remove the card, although it may be difficult to grasp the card with your fingers. In this case the spring tensioner inside the slot can be used to eject the card.

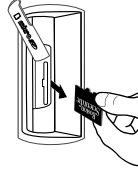

Use a fingertip to slightly press the card back into the slot, then quickly slide your finger away before the card latches back into place. Note that the card may travel some distance when it is ejected.

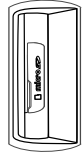

Close the microSD™ slot cover.

Remove microSD™

Repeat this procedure as needed until card is ejected.



Boston Scientific Corporation
4100 Hamline Avenue North
St. Paul, MN 55112-5798 USA

1.800.CARDIAC (227.3422)
+1.651.582.4000

www.IFU-BSCI.com



92346968-002 EN US 2021-11