

USER'S MANUAL SUPPLEMENT

LUX-Dx™

Insertable Cardiac Monitor System

REF M301, 2925, 2935

Остаряла версия. Да не се използва.
Zastaralá verze. Nepoužívat.
Forældet version. Må ikke anvendes.
Version überholt. Nicht verwenden.
Aegunud versioón. Mynn þynn xprnuðu meita.
Παλιά έκδοση. Μην την χρησιμοποιείτε.
Outdated version. Do not use.
Version obsolete. Ne pas utiliser.
Zastarjela verzija. Nemojte upotrebljavati.
Úreлт útгáfa. Notið ekki.
Versione obsoleta. Non utilizzare.
Zastarjela verzija. Neizmantot.
Úreлт útгáfa. Notið ekki.
Novcojusi versija. Nenaudokite.
Pasenusi versija. Ne pas utiliser.
Elavult verzió. Ne használja!
Utdatert versjon. Nenaudokite.
Wersja przeterminowana. Niet gebruiken.
Versão obsoleta. Não utilize.
Versione expirată. Skal ikke brukes.
Zastaraná verzia. A nu se utiliza.
Zastarela različica. Nie używać.
Vanhentunut versio. Ne se utiliza.
Föråldrad version. Använd ej.
Güncel olmayan sürüm. Kullanmayın.

These sub-sections under the **CLINIC APP** section in the **Mobile Applications** chapter of the LUX-Dx Clinician Manual have been updated.

Setup

To set up the clinic app:

1. Power on the mobile device.
2. Tap the **LUX Clinic** app icon to open the app. A screen will display, indicating that the language is set to English.
Touchscreen Tip: Gently and quickly tap the screen with your fingertip. Do not use your fingernail.
3. To continue with English as the language setting, tap **NEXT** and proceed to the next step. To change the language setting, tap **CHANGE LANGUAGE** and complete the following:
 - a. Select the preferred language by tapping the language and then tap **APPLY**. A screen will display in the selected language indicating the language you selected.
 - b. Tap **NEXT**.
4. Tap the **START SETUP** button on the **Welcome!** screen. Follow the instructions on the screen.

NOTE: If the app is not able to connect to the server using a cellular connection, the app will prompt the user to connect to Wi-Fi to complete the setup.

5. A message will confirm when the setup is complete. Tap **CONTINUE**.

Connect to Wi-Fi

Connect to Wi-Fi at any time by completing the following steps:

1. Press the Home button on the mobile device.
2. Tap the Settings icon in the upper right corner of the screen.
3. Tap **Wi-Fi**.
4. If a list of available Wi-Fi networks displays, tap to select a network from the list.
5. Follow the instructions on the screen. You may be prompted to enter a network password to connect.
6. The screen will indicate when the mobile device is connected to Wi-Fi.
7. Press the Home button on your mobile device.
8. Tap the **LUX Clinic** icon to return to the app.

Secure the mobile device screen

On a Boston Scientific mobile device, there is an option to lock the mobile device screen to prevent unauthorized users from accessing the clinic app:

1. Press the Home button on the mobile device.
2. Tap the Settings icon in the upper right corner of your screen.
3. Tap **Screen Lock**.
4. Tap to select the method to be used (e.g., PIN, Password) and follow the instructions on the screen.
5. Press the Home button on the mobile device.
6. Tap the **LUX Clinic** icon to return to the app.

Change language setting

On a Boston Scientific mobile device, the language setting is selected during app setup. Change the language setting at any time using the following steps:

1. Press the Home button on the mobile device.
2. Tap the Settings icon in the upper right corner of the screen.
3. Tap **Language** on this screen and on the next screen.
4. Select the preferred language by tapping the language and then tap **APPLY**. A screen will display in the selected language indicating the language you selected.
5. Press the Home button on the mobile device.
6. Tap the **LUX Clinic** icon to return to the app.

Остаряла версия. Да не се използва.
Zastaralá verze. Nepoužívat.
Forældet version. Må ikke anvendes.
Version überholt. Nicht verwenden.
Aegunud versioon. Ärge kasutage.
Παλιά έκδοση. Μην την χρησιμοποιείτε.
Outdated version. Do not use.
Version obsoleta. Ne utilizar.
Zastarjela verzija. Nemojte upotrebljavati.
Úreлт útгáфа. Notið ekki.
Versione obsolete. Non utilizzare.
Pasenusi versija. Neizmantot.
Elavult verzió. Ne használja!
Wersja przeterminowana. Niet gebruiken.
Versão obsoleta. Não utilize.
Dit is een verouderde versie. Niet gebruiken.
Versione expirată. Skal ikke brukes.
Zastaraná verzia. Nepoužívať.
Zastarela različica. Ne uporabite.
Vanhentunut versio. Älä käytä.
Föråldrad version. Använd ej.
Güncel olmayan sürüm. Kullanmayın.



Boston Scientific Corporation
4100 Hamline Avenue North
St. Paul, MN 55112-5798 USA

EC REP

Guidant Europe NV/SA; Boston Scientific
Green Square, Lambroekstraat 5D
1831 Diegem, Belgium

1.800.CARDIAC (227.3422)
+1.651.582.4000
www.bostonscientific.com

AU REP

Boston Scientific (Australia) Pty Ltd
PO Box 332
BOTANY NSW 1455 Australia
Free Phone 1 800 676 133
Free Fax 1 800 836 666

CE 2797

© 2023 Boston Scientific Corporation or its affiliates.
All rights reserved.

51676508-001 EN EU

2023-06

