Seal™

Single Use Biopsy Valve

Directions for Use	2
Instrucciones de uso	4
Mode d'emploi	6
Gebrauchsanweisung	8
Istruzioni per l'uso	10
Gebruiksaanwijzing	12
Instruções de Utilização	14

50676680-01 Rev. B

2020-01

Seal™

Single Use Biopsy Valve

FOR SINGLE USE ONLY

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

REUSE WARNING

For single use only. Do not reuse, reprocess or sterilize. Reuse, reprocessing or sterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or sterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

DEVICE DESCRIPTION

The Seal Single Use Biopsy Valve is an endoscopic biopsy valve that fits on the working channel port of endoscopes to facilitate device passage and maintain insufflation during endoscopic procedures. The attached cap should remain closed during the endoscopic procedure unless larger devices need to be passed.

The Seal single use biopsy valve is used to cover the opening to the biopsy/suction channel inlet of Olympus™ and Fujinon™ G5 series gastrointestinal endoscopes.

Contents

Package contents include: (1) Seal Single Use Biopsy Valve

INDICATIONS FOR USE

The Seal Single Use Biopsy Valve provides access for endoscopic device passage and exchange, helps maintain insufflation, minimizes leakage of biomaterial from the biopsy port throughout the endoscopic procedure, and provides access for irrigation.

Clinical Benefit Statement

The clinical benefit of the Seal Single Use Biopsy Valve is to maintain insufflation pressure while permitting devices to pass through the endoscope to treat the patient.

CONTRAINDICATIONS

Contraindications include those specific to any endoscopic procedure.

WARNINGS

- When irrigation is used, ensure proper techniques to avoid aspiration in the patient.
- Do not use a sharp or pointed object to prime the biopsy valve prior to use.
- Do not leave a device hanging from the valve. Doing so can cause the creation of a larger valve slit / hole that can compromise leak management.
- If the lid of the Seal Single Use Biopsy Valve is opened while attached to the endoscope during a procedure, scope suction will be compromised and chances of leakage increase. Gauze should be used to cover the biopsy valve if the lid must be opened for any reason.
- Exposure to bodily fluids may occur during connection or disconnection of these devices; adherence to Body Substance Isolation protocols is the responsibility of the user.

ADVERSE EVENTS

None known.

HOW SUPPLIED

The device is supplied non-sterile and intended for single use only. The packaging and device should be inspected prior to use.

Do not use if package is opened or damaged.

Do not use if labeling is incomplete or illegible.

Handling and Storage

This product has no special handling or storage requirements.

OPERATIONAL INSTRUCTIONS

Caution: The biopsy valve cannot be used with ultrasonic endoscopes with UC in the model number, or with dual channel endoscopes with 2T in the model number.

The Seal Single Use Biopsy Valve is designed to allow passage of a 3.2 mm device with a closed cap and once the device is removed to seal securely.

1. Remove the Seal Single Use Biopsy Valve from the pouch.
2. Place the Seal Single Use Biopsy Valve on the endoscope channel inlet.
3. Insert devices as needed through the Seal Single Use Biopsy Valve by aligning the device with the endoscope inlet and advancing slowly in a straight manner.

Caution: If the diameter of the accessory is greater than 3.2 mm open the cap for instrumentation.

4. Remove and discard the Seal Single Use Biopsy Valve after each procedure prior to reprocessing the endoscope.
5. After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

Disposal

To minimize the risk of infection or microbial hazards after use, dispose device and packaging as follows:

After use, device may contain biohazardous substances. The device and packaging should be treated and disposed of as biohazardous waste or have them treated and disposed of in accordance with any applicable hospital, administrative, and/or local government regulations. Use of a biohazardous container with biological hazard symbol is recommended. Untreated biohazardous waste should not be disposed of in the municipal waste system.

Post-Procedure

Any serious incident that occurs in relation to this device should be reported to the manufacturer and relevant local regulatory authority.

WARRANTY

For device warranty information, visit (www.bostonscientific.com/warranty).

EU Importer: Boston Scientific International B.V., Vestastraat 6, 6468 EX Kerkrade, The Netherlands

Seal is a registered trademark of Boston Scientific Corporation or its affiliates.

All other trademarks are the property of their respective owners.

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Fabricagedatum

Data de fabrico

Medical Device under EU Legislation

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da UE

EC**REP****Authorized Representative
in the European Community**

Boston Scientific Limited
Ballybrit Business Park
Galway
IRELAND

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**Manufacturer**

Manufactured for:
Boston Scientific Corporation
300 Boston Scientific Way
Marlborough, MA 01752
USA
USA Customer Service +1-888-272-1001

**Do not use if package
is damaged.****Recyclable
Package**

© 2020 Boston Scientific Corporation or its affiliates.
All rights reserved.