


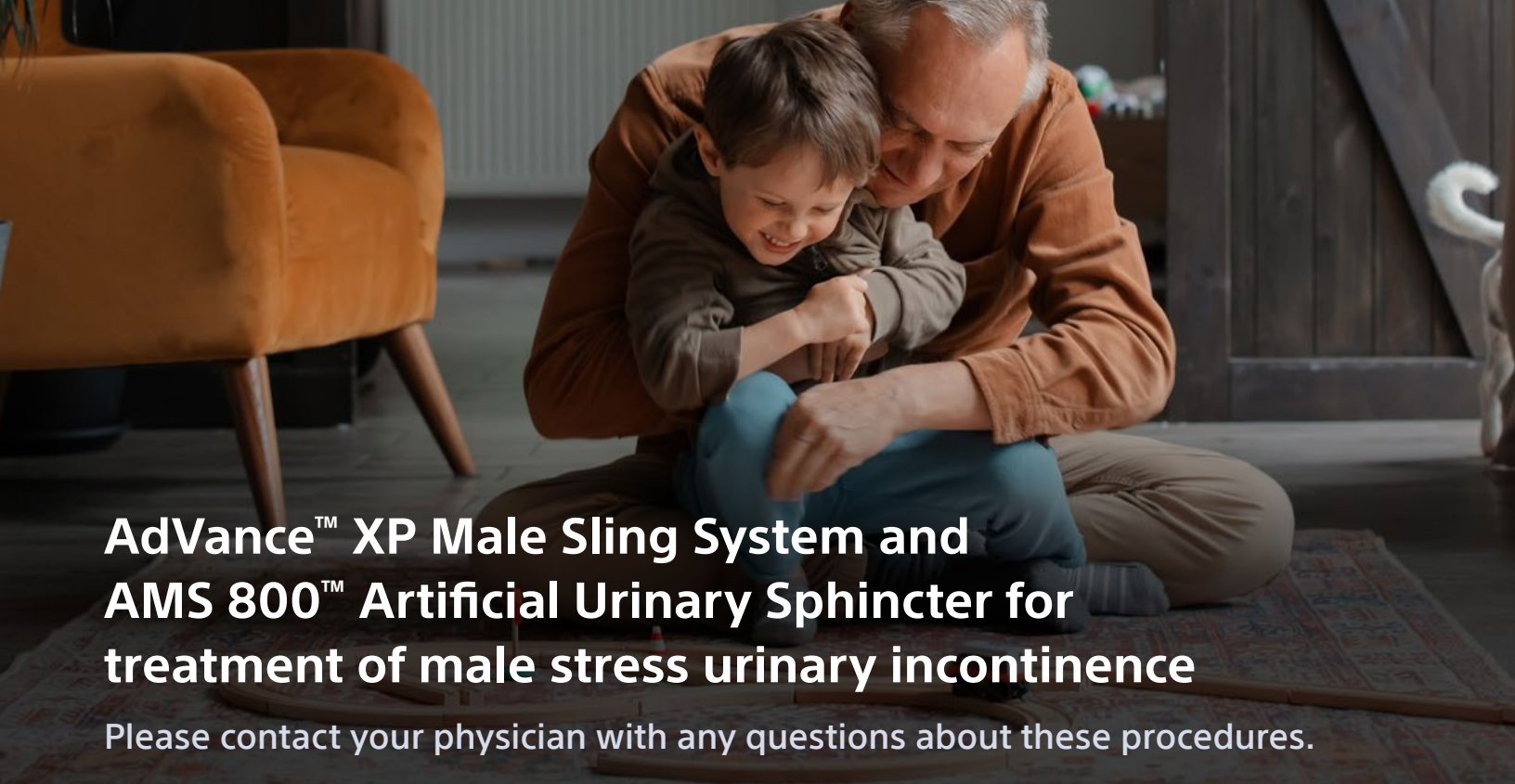

AdVance™ XP Male Sling System and AMS 800™ Artificial Urinary Sphincter for treatment of male stress urinary incontinence

Please contact your physician with any questions about these procedures.

Frequently Asked Questions (FAQ) about your procedure

How long does it take to perform the AMS 800 or AdVance XP procedures?

The procedures for each treatment usually take 1–2 hours. The procedures are generally performed in an outpatient setting, which means patients go home the same day or early the following morning.

Will I be continent after receiving the AMS 800 or AdVance XP treatment?

For the AMS 800, long-term studies show 60% to 96% of patients use 1 or no pads per day.¹⁻³ One study found that patients may be continent immediately following their AdVance XP procedure.⁴ Long-term data shows that after having the AdVance XP procedure, cured and improved rates range from 63.2% to 83% at 5 years in patients with mild to moderate incontinence.^{5,6}

Will anyone be able to visually tell I have an implant?

Once in place, your implant will be completely undetectable in appearance to others. It's fully concealed in the body.⁵ No one will know unless you tell them—even in the locker room.

How long is recovery?

Recovery depends upon whether a sling or an artificial urinary sphincter (AUS) has been implanted. With the sling, patients may resume normal activities at the physician's discretion, often within one to two weeks following the procedure.⁸ Your urologist can give you specific instructions regarding your recovery.

For those with an AUS, your urologist will manage your care following your procedure and over the following several weeks. At the four- to six-week follow-up appointment, your urologist will activate the AUS and teach you how to use the AUS device.

Common questions to ask your physician

What do I need to do before my procedure?

What are the risks of the surgery?

How satisfied are patients with the results of
this procedure?

What are some possible side effects?

Will I be able to drive myself home
after the procedure?

What do I need to do for proper healing?

Additional resources

Living with a device

You can speak to a real patient who has been treated for urinary incontinence to ask candid questions about their experience.

Call +1 (844) 433-2873 or email

MHPatientEducation@bsci.com, and a patient educator will connect you.

Identification cards

Identification cards include information about your device and how to use it. They're provided in the information packet that you'll receive from your doctor after the procedure. To request a new or replacement identification card for your Boston Scientific device, visit **BostonScientific.com/ID-Card** or call our team at +1 (844) 433-2873.

AMS 800™ Artificial Urinary Sphincter

Caution: U.S. Federal law restricts this device to sale by or on the order of a physician.

Your doctor is your best source for information on the risks and benefits of the AMS 800™ Artificial Urinary Sphincter. Talk to your doctor for a complete listing of risks, warnings and important safety information.

The AMS 800™ Artificial Urinary Sphincter is intended for use in the treatment of male stress urinary incontinence (intrinsic sphincter deficiency) following prostate surgery. Men with diabetes, spinal cord injuries or skin infections may have an increased risk of infection. Some AMS 800 devices contain an antibiotic (InhibiZone™ Antibiotic Surface Treatment). The device may not be suited for patients who are allergic to the antibiotics contained within the device (rifampin, minocycline or other tetracyclines) or have systemic lupus.

Potential risks may include: device malfunction/failure leading to additional surgery, wearing away/loss of tissue (device/tissue erosion), inability to urinate (urinary retention), infection, and pain/soreness. MH-545611-AB

AdVance™ XP Male Sling System

Your doctor is your best source for information on the risks and benefits of the AdVance™ XP Male Sling System. Talk to your doctor for a complete listing of risks, warnings and important safety information.

The AdVance™ XP Male Sling System is intended for the treatment of male stress urinary incontinence (SUI). Potential risks may include inability to urinate (urinary retention), return to incontinence, infection, erosion, device migration, pelvic organ disfunction, bleeding, and pain. These devices are contraindicated for patients with urinary tract infections, blood coagulation disorders, a compromised immune system or any other condition that would compromise healing, with renal insufficiency, and upper urinary tract relative obstruction. MH-557011-AB



Visit **FixIncontinence.com** for more information on male Stress Urinary Incontinence and to hear real patient stories.

FIXINCONTINENCE.COM
SOLUTIONS FOR MALE BLADDER LEAKAGE

References

1. Suh YS, Ko KJ, Kim TH, Sung HH, Lee KS. Long-term outcomes of primary implantation and revisions of artificial urinary sphincter in men with stress urinary incontinence. *Neurourol Urodyn.* 2017;36(7):1930-1937.
2. Deruyver Y, Schillebeeckx C, Beels E, De Ridder D, Van der Aa F. Long-term outcomes and patient satisfaction after artificial urinary sphincter implantation. *World J Urol.* 2022;40(2):497-503.
3. Sayedahmed K, Olanas R, Kaftan B, et al. Impact of previous urethroplasty on the outcome after artificial urinary sphincter implantation: a prospective evaluation. *World J Urol.* 2020;38(1):183-191.
4. Welk BK, Herschorn S. The male sling for post-prostatectomy urinary incontinence: a review of contemporary sling designs and outcomes. *BJU Int.* 2012 Feb;109(3):328-44.
5. Mumm JN, Klehr B, Rodler S, et al. Five-year results of a prospective multicenter trial: AdVance XP for postprostatectomy-incontinence in patients with favorable prognostic factors. *Urol Int.* 2021;105:421-427.
6. Rizvi IG, Ravindra P, Pipe M, Sohawon R, King T, Belal M. The AdVance™ male sling: does it stand the test of time? *Scand J Urol.* 2021;55:155-160.
7. Data on file with Boston Scientific.
8. AdVance™ XP Male Sling System Directions for Use. Boston Scientific. 2020.

FixIncontinence.com is a website sponsored by Boston Scientific.

All images are the property of Boston Scientific. All trademarks are the property of their respective owners.

CONTENT IS PROVIDED BY BOSTON SCIENTIFIC. BOSTON SCIENTIFIC IS DEDICATED TO TRANSFORMING LIVES THROUGH INNOVATIVE MEDICAL SOLUTIONS THAT IMPROVE THE HEALTH OF PATIENTS AROUND THE WORLD.

**Boston
Scientific**
Advancing science for life™

Boston Scientific Corporation
300 Boston Scientific Way
Marlborough, MA 01752-1234
www.BostonScientific.com

©2023 Boston Scientific Corporation
or its affiliates. All rights reserved.

MH-496805-AC OCT 2023